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SERVOX Soft Tube 22 *connect*

1. General description

The SERVVOX Soft Tube 22, fenestrated or unfenestrated, consists of a cannula tube which is inserted into the trachea and a funnel-shaped attachment with an internal diameter of 22 mm at its outer end. The cannula tube is fenestrated or unfenestrated depending on the version. In addition, the neck flange features fastening eyelets for attaching a cannula fixation strap.

The SERVVOX Soft Tube 22 *connect* consists of a cannula tube which is inserted into the trachea and an attachment with an internal diameter of 22 mm at its outer end. A connector is fitted at the upper end of the cannula tube which is used to insert the SERVVOX Soft Tube 22 *connect* into a base plate (e.g., SERVVOX base plate), enabling the tube to be secured without the use of a cannula fixation strap.

The SERVVOX Soft Tube 22 is a high-quality product made of medical-grade silicone and is supplied in non-sterile packaging. (The materials used comply with relevant national and international standards.) The SERVVOX Soft Tube 22 is latex- and PVC-free, and does not contain any metallic materials, which means that there are no restrictions in the use of magnetic resonance imaging (MRI).

Thanks to its material properties and high-quality workmanship, the SERVVOX Soft Tube 22 is a tracheal tube which is well tolerated and can be used over a long period.

The silicone adapts optimally to the anatomical conditions in the trachea and therefore ensures a high level of wearing comfort.

The 22 mm attachment at the outer end is used to attach a heat and moisture exchanger (HME), a tracheostoma valve or a speaking valve with a 22 mm connector.

The SERVVOX Soft Tube 22 is a single-patient-use product.

2. Indication

The SERVVOX Soft Tube 22 can be used for patients after tracheotomy or laryngectomy.

The SERVVOX Soft Tube 22 is inserted into an artificially created tracheostoma and thus ensures that the tracheostoma is stabilised and kept open.

3. Contraindications

We advise against the use of the SERVVOX Soft Tube 22 in the event of known allergies to silicone.

The use of alternative products is recommended if a tracheostoma is too narrow or inadequately positioned.

Laryngectomy patients should **never** insert speaking valves, which are only intended for patients who have undergone a tracheotomy, into the 22 mm attachment, as this can lead to **shortness of breath or suffocation**.

4. Side effects

The following side effects may occur when using the SERVVOX Soft Tube 22:

- Contamination of the surrounding tissue by unavoidable colonisation of the products by e.g., yeast fungi (Candida), bacteria or similar (see Point 5: Cleaning and care)
- Possible formation of granulation tissue with the fenestrated model
- Possible blockage of the fenestration if there is strong secretion or if secretion has a tough consistency

If you have any questions or if complications arise, please consult your attending doctor.

5. Use

Caution: Please do not use any oily substances or silicone spray as lubricants, since these may change the material properties of the silicone.

We recommend cleaning the SERVVOX Soft Tube 22 in a suitable cleaning solution before inserting it.

Prior to insertion, please make sure that you have selected the correct size. When removing the product from the packaging, please also ensure that the packaging is unopened and intact. Check the product for any damage before inserting it.

Note: The SERVVOX Soft Tube 22 *connect* is supplied unfenestrated. Fenestrations can be customised at a later stage to accommodate the individual patient's anatomy. Any such subsequent fenestration must in all cases only be performed by authorised personnel or your attending doctor. Fenestration must be performed outside the tracheostoma or the trachea.

a. Inserting the cannula (SERVOX Soft Tube 22 fenestrated, unfenestrated):

Prior to inserting the SERVOX Soft Tube 22 into the tracheostoma, ensure that the cannula can slide in without difficulty.

Attach the SERVOX Soft Tube 22 using a cannula fixation strap. The fixation strap must be positioned so that it does not restrict the mobility of the head and the SERVOX Soft Tube 22 is held firmly in the tracheostoma.

Please only use recommended cannula fixation straps. Cannula fixation straps with sharp-edged hooks can damage the fastening eyelets. We recommend using the SERVOX Tracheo Tape *soft* cannula strap.

b. Inserting the cannula (SERVOX Soft Tube *connect*):

Prior to inserting the SERVOX Soft Tube 22 *connect* into the tracheostoma, ensure that the cannula can slide in without difficulty.

Fix a base plate (e.g., SERVOX base plate) around your tracheostoma. Please observe the corresponding instructions for use.

Now insert the SERVOX Soft Tube 22 *connect* through the 22 mm opening of the base plate into the tracheostoma. The SERVOX Soft Tube 22 *connect* has been correctly inserted if it has engaged in the attachment ring of the base plate.

c. Cleaning and care:

Instructions:

- The product must not be sterilised or exposed to direct sunlight. This can lead to destruction of the material.

- Prior to cleaning, any aids inserted into the 22 mm attachment must be removed and cleaned separately or disposed of. In doing so, please observe the recommendations of the respective manufacturer.
- To extend the life of the SERVOX Soft Tube 22, it should be cleaned at least 1-2 times per day, or even more frequently, if required. Prior to cleaning, the product must be removed from the tracheostoma.

Warning: The cleaning brush must not be inserted into the tracheostoma as this can lead to aspiration!

The SERVOX Soft Tube 22 can be heated to temperatures of up to 65°C in a water bath. The SERVOX Soft Tube 22 can be cleaned using standard mild household cleaning agents. We recommend the use of a cleaning solution for tracheostomy tubes. Rinse the SERVOX Soft Tube 22 thoroughly under running water after cleaning and dry it using a lint-free cloth.

Never use aggressive household detergents or high-concentration alcohol for cleaning, since these may alter the material properties. Furthermore, do not use any disinfectants that release chlorine, or agents that contain strong alkalis.

Stubbornly adhering secretion can be carefully removed using a cleaning brush. In doing so, avoid applying excessive pressure. In order to prevent damage to the silicone, mechanical or sharpened tools such as scrapers, knives, fingernails, etc. must not be used under any circumstances.

Following cleaning, the tubes must be stored in a dry place.

Even after being cleaned multiple times, the SERVOX Soft Tube 22 can be used on one patient several times over a period of at least 29 days and a maximum of 6 months, in accordance with the specified standards and in compliance with the application instructions.

6. Warnings:

- Do not make any changes to the product yourself!
- Any adjustments (fenestration, shortening) must only be performed by qualified specialist personnel!
- Fenestration must never be performed on an inserted cannula.
- Laryngectomy patients should never insert speaking valves, which are only intended for patients who have undergone a tracheotomy, into the 22 mm attachment, as this can lead to shortness of breath or suffocation.
- We advise against the use of the product if it shows signs of damage such as sharp edges, tears or similar.

Failure to follow these instructions can have serious consequences for the health of the patient and may even result in life-threatening injuries. The manufacturer does not accept any liability for damage resulting from manipulation or modification of the product.

7. Safety

Each product is assigned a batch number, which is shown on the outer packaging and on the label, in order to ensure unambiguous traceability. The batch number must be noted on the patient card supplied or the enclosed label must be attached to the card.

ENGLISH

EN

We recommend that patients should carry their patient card with them at all times.

Order Information

Description			SERVOX Soft Tube 22 non-fenestrated	SERVOX Soft Tube 22 fenestrated	SERVOX Soft Tube 22 connect
Size	OD*	ID*	Length*	REF	REF
8	12	9,5	27	26472	-
8	12	9,5	36	26473	27376
8	12	9,5	55	26474	27377
9	13,5	10,5	27	26475	-
9	13,5	10,5	36	26476	27378
9	13,5	10,5	55	26477	27379
10	15	12	27	26478	-
10	15	12	36	26479	27380
10	15	12	55	26480	27381
12	17	13,5	27	26481	-
12	17	13,5	36	26482	27382
12	17	13,5	55	26483	27383

OD: Outer Diameter at the cannula tip

ID: Inner Diameter at the cannula tip

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Orbisana Healthcare GmbH