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SERVOX Elasto Tube *standard | voice | vent*

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SERVOX Elasto Tube *standard | voice | vent*

1. General Description

The SERVOX Elasto Tube *standard* consists of an outer cannula and two inner cannulas.

The SERVOX Elasto Tube *voice* consists of a fenestrated outer cannula, two inner inner cannulas (fenestrated and non-fenestrated) with 15 mm standard connector and a SERVOX Elasto Tube speaking valve.

The SERVOX Elasto Tube *vent* consists of an outer cannula, two inner cannulas with 15 mm standard connector and a SERVOX Elasto Tube clip for additional fixation of the inner cannula, e.g. for ventilated patients.

The **outer tube** consists of a cannula tube which is inserted in the windpipe (trachea) and a neck flange with a lock for the inner cannulas. The eyelets of the neck flange offer a firm hold of cannula due to a tracheostomy tube holder. Depending on the model, the cannula is supplied fenestrated or not.

The **inner cannula** is a tube that can be inserted in the outer cannula of the SERVOX Elasto Tube. The inner cannula is secured using the hook present on the outer cannula and has got a flexible 15 mm standard connector. Depending on the model this tube is either fenestrated or non-fenestrated.

The SERVOX Elasto Tube is a quality product made from medical-grade polyurethane and is supplied in non-sterile form.

The SERVOX Elasto Tube is not made from natural rubber latex, free from PVC and does not contain any metallic materials, so that there are no restrictions when magnetic resonance imaging (MRI) is used.

Its material properties and good processing make the SERVOX Elasto Tube a highly compatible tracheal cannula which can be used in the long term.

The polyurethane adapts to body temperature, to the anatomical conditions in the trachea, guaranteeing a high level of wearing comfort.

Medical aids with a 15 mm standard connection (e.g. SERVOX Elasto Tube speaking valve, artificial nose) can be attached on the 15 mm standard connector.

The SERVOX Elasto Tube suitable for a single patient only.

2. Indications

The SERVOX Elasto Tube can be used for patients after a tracheotomy or laryngectomy to ensure that the tracheostoma is stabilised and kept open.

3. Contraindications

In the case of known allergies to polyurethane, it is advised not to use the SERVOX Elasto Tube.

In the case of an excessively narrow or unsuitably positioned tracheostoma, the use of alternative products is recommended.

Under no circumstances may patients who have undergone laryngectomy fit the 15 mm standard connector with speech valves intended for patients who have undergone tracheotomy because this may lead to **dyspnoea or suffocation!**

4. Side Effects

The following side effects may arise when the SERVOX Elasto Tube is used:

- unavoidable contamination of the surrounding tissue by yeast fungi (candida), bacteria, etc. (see section on Cleaning and Care)
- formation of hematomas, particularly on first use
- possible formation of granulation tissues with fenestrated models
- possible blockage of the fenestration in case of abundant or dense secretions

In case of any questions or complications please consult your doctor.

5. Use

Before using the SERVOX Elasto Tube, we recommend cleaning in a suitable cleaning solution (see section on cleaning and care).

Before using, make sure that you have chosen the right size. In addition, when removing the product from its packaging, make sure that the latter is unopened and intact. Before using, check the product for any damage.

When fitting the SERVOX Elasto Tube speaking valve, ensure that the valve suspension (hinge) is on the top.

a. Insertion of the cannula:

Before inserting the SERVOX Elasto Tube into the tracheostoma, ensure that the tracheostomy tube is properly lubricated.

Pad the neck flange with a suitable tracheal compress.

Fix the SERVOX Elasto Tube with a tracheostomy tube holder. The tracheostomy tube holder must be positioned so that the head still has freedom of movement while still securing a firm hold of the SERVOX Elasto Tube in the tracheostoma.

b. Replacing the inner cannula

The inner cannula should be removed and cleaned when the patient notices mucus in the SERVOX Elasto Tube that cannot be removed by coughing or aspiration. During this process the cannula should be replaced with the second tube. Any medical aids attached to the 15 mm standard connector should be removed before changing the inner cannula.

To extract the inner cannula hold the outer cannula tight with one hand on the neck flange and use the other hand to grab the inner cannula ledge near the neck flange. Remove the inner cannula from the hook and extract it carefully.

Insert the spare inner cannula into the outer cannula. Secure the inner cannula behind the hook of the outer cannula.

c. Cleaning and Care

Advice:

- The product must not be sterilised.
- Do not expose the tracheostomy tube to direct sunlight, as it may lead to the deformation of the material.
- Before cleaning, any medical devices attached to the 15-mm standard connector must be disposed of or removed and cleaned separately in accordance with the manufacturer's recommendations.
- To prolong the service life of the SERVOX Elasto Tube it should be cleaned at least 1-2 times a day or more frequently if necessary. Before cleaning the product must be removed from the tracheostoma.

Warning: The cleaning brush must not be inserted in the tracheostoma as it can be aspirated and lead to suffocation.

We recommend the use of a cleaning solution for tracheal cannulas. The SERVOX Elasto Tube and its accessories may be heated to temperatures of

up to 65°C in a water bath. Cleaning of the SERVVOX Elasto Tube may be carried out with mild household cleaning products. After cleaning, rinse the SERVVOX Elasto Tube with plenty of running water and dry it with a lint-free cloth or let it dry at room temperature.

Note: Under no circumstances may cleaning be carried out with aggressive household products or high-percentage alcohol as these may alter the material characteristics. In addition, disinfectants that release chlorine must not be used, and nor may products containing strong lyes. Do not boil the tracheal cannula and its accessories!

Secretion that is stuck on can be removed carefully with a cleaning brush. When doing so, avoid applying excessive pressure. To prevent damage to the cannula, mechanical or sharp-edged aids such as scrapers, knives, fingernails, etc. should not be used under any circumstances.

After cleaning, the aid should be stored in dry conditions.

The SERVVOX Elasto Tube can be used repeatedly by one patient for at least 29 days and at most 6 months, considering all cleaning and application instructions.

6. Warnings:

- Do not modify your product in any way!
- Any adaptations, like additional fenestration or shortening may be carried out by qualified technical personnel only.
- **Under no circumstances** may patients who have undergone laryngectomy fit the 15 mm standard connector with speaking valves intended for patients who have undergone tracheotomy because this may lead to **dyspnoea or suffocation!**
- If the product presents any damage such as sharp edges or cracks do not use it and proceed to return to your provider.

Non-observance of these instructions can have serious effects on the health of the patient and cause life-threatening complications. The manufacturer does not accept any liability for any damage resulting from non-observance of these instructions or from a manipulation of the product.

7. Safety

The batch number of each tube is shown on both the packaging and label to ensure that it can be traced back to its origin. This number should be entered in the supplied ID card that should always be carried by the patient.

8. Ordering Advice:

The individual **SERVOX** Elasto Tube components are to be ordered separately; please request further information on this.

Every **SERVOX** Elasto Tube inner cannula is usable in each **SERVOX** Elasto Tube outer cannula of the same size!

Order Information

Sets

Dimensions						SERVOX Elasto Tube standard	SERVOX Elasto Tube voice	SERVOX Elasto Tube vent
Size	ED* (a)	ED* (b)	ID* (OC)	ID* (IC)	Length* (L)	REF	REF	REF
7,0	10,6	9,2	7,8	5,6	68	33025	33031	33061
8,0	12,5	11,0	8,8	6,2	68	33026	33032	33062
9,0	13,6	12,1	9,8	7,1	82	33027	33033	33063
10,0	14,6	13,0	10,8	8,1	82	33028	33034	33064
11,0	15,3	14,0	11,8	9,1	89	33029	33035	33065
12,0	16,5	15,0	12,8	10,3	89	33030	33036	33066

ED (a): external diameter of the outer cannula behind the neck plate

ED (b): external diameter at the tip of the cannula

ID (OC): smallest internal diameter of the outer cannula

ID (IC): smallest internal diameter of the inner cannula

Length (L): from neck plate to cannula tip along the outside bend

* Data in mm

Order Information

Accessories

Description	Variante	Bestell-Nr.
SERVOX Elasto Tube Clip	---	28390
SERVOX Elasto Tube Sprechventil	---	25930
SERVOX Tracheo Tape M duo	white, Velcro fastener	35207

EN

Single components - Outer cannula

Size	Version	Order-Nr.	Version	Order-Nr.
7	non-fenestrated	33037	fenestrated	33043
8	non-fenestrated	33038	fenestrated	33044
9	non-fenestrated	33039	fenestrated	33045
10	non-fenestrated	33040	fenestrated	33046
11	non-fenestrated	33041	fenestrated	33047
12	non-fenestrated	33042	fenestrated	33048

Single components - Inner cannula

Size	Version	Order-Nr.	Version	Order-Nr.
7	non-fenestrated	33049	fenestrated	33055
8	non-fenestrated	33050	fenestrated	33056
9	non-fenestrated	33051	fenestrated	33057
10	non-fenestrated	33052	fenestrated	33058
11	non-fenestrated	33053	fenestrated	33059
12	non-fenestrated	33054	fenestrated	33060

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