SERVOX[®] digital XL

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Speech Aid Complete Set

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INSTRUCTIONS FOR USE

SERVOX® digital XL Speech Aid Complete Set

Symbols used

182

Contents

1 Foreword	15
1.1 Intended purpose	15
1.2 Indication	15
1.3 Contraindication	15
1.4 Side effects	15
2 Safety instructions	15
3 Scope of delivery	17
4 Description	17
4.1 Product overview	17
5 Application	17
5.1 Readiness for use and deactivation of the SERVOX® digital XL speech aid	17
5.2 Removing and inserting batteries	18
5.3 Battery status indicator	18
5.4 Removing or sliding on the titanium sleeve	18
5.5 Reading the serial number	19
5.6 Using the buttons	19
5.7 Setting the audio frequency	19
5.8 Fine tuning the pitch	20
5.9 Adjusting the volume	20
5.10 Application points	20
5.11 Notes on speech formation	20
6 Troubleshooting	21
7 Accessories	21
7.1 SERVOX® digital XL screw caps hard tone and soft tone"	21
7.2 SERVOX® digital XL oral tube with adapter	21
7.3 SERVOX® digital XL strap set	21
7.4 SERVOX® digital XL belt pouch	22
7.5 Batteries, charger and travel adapter (optional)	22
7.6 SERVOX® digital XL software/SERVOX® digital XL data cable (can be ordered)	22
8 Cleaning/disinfection	22
8.1 Methods used	22
8.2 SERVOX® digital XL speech aid and SERVOX® digital XL screw caps	22
8.3 SERVOX® digital XL oral tubes and adapter	23
8.4 SERVOX® digital XL belt pouch and SERVOX® digital XL strap set	23
9 Maintenance and repair	23
10 Operation, storage and transport	23
11 Period of use and disposal	23
12 Technical data	24
12.1 Information on electromagnetic compatibility (EMC)	24
13 Order information	25
14 Disclaimer	25

1 Foreword

Dear customer,

We would like to thank you for your trust and the purchase of our quality product.

The SERVVOX® digital XL speech aid complete set will be referred to as the SERVVOX® digital XL in the further course of this instruction manual. This instruction manual is intended for patients, relatives and medical staff.

The SERVVOX® digital XL can be used in the home and clinical setting by patients from about 2.5 years of age. Children between 2.5 and 12 years of age or patients with cognitive disorders should only use the SERVVOX® digital XL under the supervision or with the support of relatives or healthcare professionals. Use of the SERVVOX® digital XL can be practised within the framework of logopaedic care.

Note

This manual does not replace comprehensive speech therapy.

The SERVVOX® digital XL must be used in line with the descriptions in this instruction manual and in accordance with the recognised rules of medicine.

Note

Read this instruction manual carefully. It contains important information on the safe and proper use of the product. Keep this instruction manual close to the device for future reference.

EN

The SERVVOX® digital XL is marked with a serial number for clear traceability. It is indicated on the type plate behind the marking **SN**. If you have any questions or problems with the SERVVOX® digital XL, please contact your field representative or Orbisana Healthcare GmbH, quoting the product's serial number.

1.1 Intended purpose

The SERVVOX® speech aids offer patients, who have lost their voice due to injury, illness or surgical removal of the larynx, the opportunity to quickly regain the ability to communicate.

The SERVVOX® speech aids can also be used by patients who are temporarily voiceless due to a tracheotomy, who have to protect their vocal folds due to illness or who are connected to a respirator.

Even patients who are comfortable using the oesophageal voice can use the SERVVOX® speech aids in a variety of scenarios, e.g. for making phone calls, at business meetings, when they are tired, sick or very stressed, in an emergency, or in settings where speaking in a loud voice is essential.

The SERVVOX® digital XL may only be used for the applications listed above. The manufacturer assumes no liability for all other applications and resulting damages of any kind.

Note

SERVVOX® digital XL and the accessories contained in it, such as screw caps or oral tubes with adapters, are single-patient products and must not be reused on additional persons. There is a risk of infection.

1.2 Indication

- After a laryngectomy
- After a tracheotomy
- For ventilated patients
- For patients who have lost their voice due to injury or illness
- For patients who need to rest their vocal folds due to illness
- As an aid for laryngectomised patients who use ructus speech or a shunt valve for voice production

1.3 Contraindication

- With surgical wounds on the neck or cheek area
- During radiotherapy in the neck area

In these situations, the speech aid can be used in combination with the SERVVOX® digital XL oral tube after consultation with the attending physician.

1.4 Side effects

If the indication, contraindication, safety instructions and instructions for use are observed, no side effects are known.

2 Safety instructions

Safe and correct use of the SERVVOX® digital XL is important for your health. Ignoring these safety instructions can have significant effects on your health or even cause life-threatening injuries.

Attention: The word "Attention" indicates a situation that can cause damage to your product if not avoided.

In the following, warnings on the safe use of the product are indicated with the  symbol.

Warning: The word "Warning" indicates a situation that, if not avoided, can result in grave injury or even death.

Caution: The word "Caution" indicates a situation that can result in minor or moderate injury if not avoided.

Please observe the following safety instructions:

Warning

Risk of injury: Children can swallow small parts and choke, strangle themselves with the carrying strap, damage their hearing or burn themselves.

- Do not leave children unsupervised with the SERVVOX® digital XL!

Risk of injury/infection: If you place the SERVVOX® digital XL speech aid on injured or inflamed skin, you may injure or infect yourself.

- Only place the speech aid on uninjured, healthy skin!

Risk of injury/infection: If you place objects on a tracheostoma, breathing will be restricted and you may get injured or infected.

- Never place the SERVVOX® digital XL speech aid on a tracheostoma!

Risk of fire/explosion: The SERVVOX® digital XL speech aid can produce sparks and possibly ignite explosive gases.

- Do not use the speech aid at petrol stations or in the presence of flammable gases, such as oxygen or anaesthetic gases!
- In areas with explosive gases, always turn the bottom cover loose!

Risk of fire/explosion: The SERVVOX® digital XL speech aid can be operated with normal batteries and rechargeable batteries, but only rechargeable batteries can be charged. Normal batteries can explode when connected to a charger.

- Never insert normal batteries into the charger!

Risk of injury: The volume of the SERVVOX® digital XL speech aid can reach approx. 87 dB at the highest setting. It can cause temporary or permanent hearing damage if you hold it directly to an ear.

- Never hold the speech aid directly to an ear or place it on top of it!

Risk of injury: Users can strangle themselves with the carrying strap.

- Always wear the SERVVOX® digital XL speech aid in front of the body in a controlled manner!
- Make sure that speech aid or strap cannot get caught or jammed!

Risk of injury: If the SERVVOX® digital XL speech aid is used continuously at full volume, it may become hot. You may burn yourself on it.

- Turn the bottom cover loosely and let the speech aid cool down if it gets hot (over 48°C)!
- Do not leave children unattended with the speech aid!

Caution

Risk of infection: If the SERVVOX® digital XL speech aid or its accessories are used by more than one person, cross-infections may occur.

- Do not give the speech aid or accessories to other people!

Risk of electric shock: If the SERVVOX® digital XL battery holder is turned too hard or too fast when inserting the battery holder, a short circuit may occur. You may receive an electric shock.

- Insert the battery holder carefully and slowly!

Attention

Moisture: The SERVVOX® digital XL speech aid is not waterproof! If the inside of the speech aid gets wet, the speech aid may be irreparably damaged.

- Never immerse the speech aid in liquids!
- If possible, do not place or lay the speech aid next to drinks or meals so that it does not accidentally become soiled or soaked!

Damage due to fall or impact: The bottom cover is the heaviest point of the SERVVOX® digital XL speech aid. Especially when the speech aid is worn around the neck, it can hit hard surfaces with the bottom cover and be damaged. If the speech aid is dropped, individual parts can tear on impact.

- Always wear the speech aid in front of the body in a controlled manner!
- Hold the speech aid firmly and always put it down so that it cannot fall down!
- Check the speech aid after a fall and have it checked by Orbisana Healthcare GmbH if necessary!

Repairs and maintenance: The SERVVOX® digital XL speech aid may only be serviced or repaired by specialist personnel authorised by Orbisana Healthcare GmbH.

- Do not repair the speech aid yourself!

Accessories and spare parts: The accessories are adapted for use with the SERVVOX® digital XL speech aid. Non-approved accessories may impair the functions or damage the speech aid.

- Only use the speech aid with the approved accessories!
- Only use approved accessories and spare parts from Orbisana Healthcare GmbH!
- Commercially available batteries must comply with the specifications in the technical data!
- Do not use defective or damaged batteries!

Note

Please inform us or the responsible authorities if you experience a life-threatening illness or injury or permanent physical damage as a result of using our product, or require hospital treatment or have to extend hospital treatment due to using our product.

3 Scope of delivery

- SERVOX® digital XL speech aid (Art. No. 37270)
- SERVOX® digital XL screw cap soft tone
- SERVOX® digital XL screw cap hard tone (preassembled)
- SERVOX® digital XL oral tube with adapter (optional)
- SERVOX® digital XL strap set
- SERVOX® digital XL belt pouch
- SERVOX® digital XL battery holder (2 x)
- 8 AAA rechargeable batteries (optional)
- Charger Varta LCD plug with instructions for use (commercially available) (optional)
- kwmobile universal travel adapter (commercially available) (optional)
- SERVOX® digital XL instruction manual

4 Description

The SERVOX® digital XL consists of the SERVOX® digital XL speech aid (Art. No. 37270) and the accessories described in the scope of delivery. The SERVOX® speech aid is an electrically operated handset made of plastic, with a sleeve made of titanium. The SERVOX® digital XL screw cap is screwed onto the upper end of the speech aid and contains a transducer and a diaphragm for transmitting vibrations.

Activated by pressing a button, this transducer vibrates to produce a sound. If the speech aid is pressed with the screw cap onto one of the application points on the cheek, base of the mouth or neck, this sound is transmitted through the tissue into the mouth/throat and can be formed with speaking movements of the lips, tongue and used for speaking. Volume and sound can be adjusted via a volume control and by turning the screw cap. The speech aid can be operated either with AAA rechargeable batteries or with AAA batteries.

Patients who have lost their vocal cords or cannot use them due to injury, illness or surgical removal of the larynx (laryngectomy) can use the sounds produced by the SERVOX® digital XL speech aid to speak.

4.1 Product overview

Figure (1) Product overview

5 Application

5.1 Readiness for use and deactivation of the SERVOX® digital XL speech aid

The speech aid is ready for use as soon as charged rechargeable batteries or normal batteries are inserted and the base cover is firmly screwed on.

- ➔ **If the speech aid is to be deactivated for a short time**, turn the bottom cover approx. one turn to the left, anticlockwise. If the bottom cover is not completely screwed on, the contact to the batteries is interrupted.
- ➔ **To restore readiness for use**, turn the bottom cover back to the right clockwise. When the bottom cover is completely unscrewed, the contact to the batteries is re-established.

5.2 Removing and inserting batteries

Attention

- Always observe the information and safety instructions on the packaging of batteries!
- Do not use defective or damaged rechargeable batteries!
- Do not use rechargeable batteries and batteries at the same time!
- Do not use rechargeable batteries or batteries with different states of charge at the same time!
- If the SERVOX® digital XL speech aid will not be used for a long time, remove the batteries from the speech aid. Batteries can corrode or leak and then damage the speech aid.
- Do not dispose of batteries in the household waste!

Note

- Before first use, fully charge the rechargeable batteries. (Follow the instructions for use of the rechargeable batteries and charger!)
- Always have 4 charged spare rechargeable batteries/4 full AAA batteries ready to use the speech aid without interruption.
- Pay attention to the polarity of batteries when inserting them.

Figure (2) Removing the battery holder

Figure (3) Inserting the rechargeable batteries

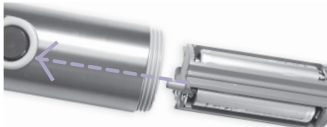
Figure (4) Aligning and inserting the battery holder

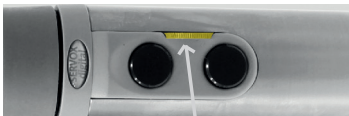
Figure (5) Battery status indicator

The battery holder of the SERVVOX® digital XL speech aid is preassembled

Removing the battery holder:

- ➔ Screw off the bottom cover (E) anticlockwise, holding the titanium cover to prevent it from slipping off the speech aid
- ➔ Pull out the battery holder (H) straight

Inserting batteries:

The battery poles are marked on the bottom of the battery holder (H)

- ➔ Insert 4 fully charged AAA rechargeable batteries/4 full AAA batteries into the battery holder with the terminals aligned with the marking on the battery holder base

Inserting the battery holder:

The battery holder has a fixing pin (I) on the top side which is inserted into the pin hole provided inside the speech aid and thus fixes the battery holder in the correct position.

- ➔ Align the battery holder (H) with the fixing pin (I) on the left edge of the buttons as in figure 4 and align the guide mark on the battery holder centrally with the buttons
- ➔ Slide the battery holder (H) straight into the speech aid, slowly and without pressure, until the fixing pin (I) slides into the pin hole
- ➔ Then, put the bottom cover back on straight and screw it on hand-tight

5.3 Battery status indicator

If the battery power decreases, the yellow LED light in the volume control/battery status indicator (C) flashes. If the battery power is no longer sufficient to maintain the normal functions, the LED light is permanently lit and no sound is produced.

- ➔ Replace batteries

5.4 Removing or sliding on the titanium sleeve

Caution

Risk of injury: The skin can be pinched when the titanium sleeve is slid off or on.

- Move the titanium sleeve slowly and carefully!

The titanium sleeve protects the electrical components of the SERVVOX® digital XL speech aid. Under the titanium sleeve there is also the type plate with the identification features of the speech aid and the removable strap protector for attaching the carrying strap.

Figure (6) Titanium sleeve

Figure (8) Type plate with all identification characteristics

Removing or sliding on the titanium sleeve

- ➔ Screw off the bottom cover and remove the battery holder (H)
- ➔ Hold the speech aid by the screw cap and pull the titanium cover downwards or push it on from below.
- ➔ Replace the battery holder and screw the bottom cover back on

Figure (9) Strap protector for attaching the carrying strap

Figure (7) Bottom cover

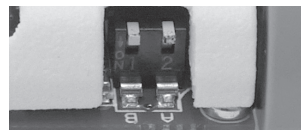
Figure (10) 2 DIP switches for setting the audio frequency

5.5 Reading the serial number

The serial number is located on the type plate of the SERVVOX® digital XL speech aid. To read the serial number, unscrew the bottom cover and slide the titanium sleeve off the housing.

On the plastic housing underneath, the serial number is indicated on the type plate behind the marking "SN".

Figure (11)
Serial number strap

5.6 Using the buttons

In natural speech, individual syllables, words or parts of sentences are emphasised by a change in pitch. The SERVVOX® digital XL speech aid therefore has 2 buttons that are assigned different pitches.

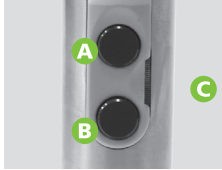
Figure (12) Control elements

Figure (13) Operating the buttons by tilting the thumb

Figure (14) Operating the buttons with the index and middle finger

- A** Button A **B** Button B **C** Volume control (wheel)

In the factory setting, the upper button (A) is preset in a lower audio frequency as the basic tone for speaking and the lower button (B) in a higher audio frequency for emphasising individual syllables, words or parts of sentences. The pitch of each button can be adjusted individually.

- Hold button (A) to speak (basic tone)
- Briefly change to button (B) to emphasise individual language elements

The buttons can be operated with the index and middle fingers or by tilting the thumb (see Figure 13 and 14).

5.7 Setting the audio frequency

Warning

- Never hold the SERVVOX® digital XL speech aid to your ear!
- Never adjust the SERVVOX® digital XL speech aid while holding it at an application point!

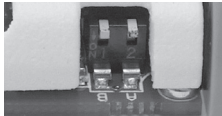
Figure (15) DIP switches

Figure (16) Setting

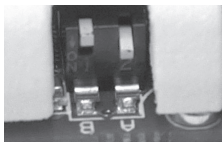
Figure (17) Setting button A

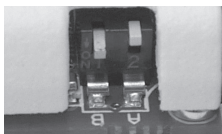
Figure (18) Setting button B

To make the speech sound as natural as possible, the pitch should be adjusted as close as possible to the former voice position.

Setting the audio frequency of the buttons

First remove the titanium sleeve. Replace the battery holder and screw the base cover back on. There are 2 DIP switches on the back of the housing. Each of the DIP switches is assigned to one of the buttons.

The assignment "A" and "B" is labelled on the DIP switch side.

Assignment of the DIP switches

- Button A = DIP switch 2
 - Button B = DIP switch 1
- The DIP switches can be operated with a pin.

Setting options with DIP switch and button

If both DIP switches are in the OFF position (top), the audio frequency is secured against change. If the DIP switch of a button is set to ON, the sound frequency of this button can be increased or decreased by pressing the other buttons.

- Increase frequency by 1 Hz -> press button A 1 x
- Decrease frequency by 1 Hz -> press button B 1 x
- Press repeatedly to increase or decrease each in 1 HZ steps
- When a button is held longer, the value increases or decreases continuously.

Setting button A

- Slide DIP switch 2 to ON (down) position
- Slide DIP switch 1 to OFF (up) position
- Setting the audio frequency of button A using the two buttons
- Slide DIP switch 2 back to OFF (up) position to save the setting

Setting button B

- Slide DIP switch 1 to ON (down) position
- Slide DIP switch 2 to OFF (up) position
- Setting the audio frequency of button B using the two buttons
- Slide DIP switch 1 back to OFF (up) position to save and the setting

After finishing the settings, both DIP switches must be in the OFF position (top) again. Saving the settings in the speech aid processor can take up to 30 seconds. After finishing the settings, wait 30 seconds and only then unscrew the bottom cover and remove the battery holder, slide on the titanium cover from below, reinsert the battery holder and screw on the bottom cover again.

5.8 Fine tuning the pitch

By turning the screw cap (F), the pitch can still be changed slightly. Be careful not to overtighten the screw cap so as not to damage it.

- For a 'clearer sound', tighten the screw cap slightly
- For a 'purer sound' at maximum volume, loosen the screw cap slightly

5.9 Adjusting the volume

Warning

- Never hold the SERVOX® digital XL speech aid directly to an ear or place it on top of it!

Figure (19) Volume control wheel

The volume can be adjusted with the volume control wheel next to the buttons.

- Increase volume by one level -> 1 x slide the wheel upwards
- Reduce volume by one level -> 1 x slide wheel down

The maximum volume is about 87 dB.

Changing the volume for both buttons simultaneously

- Do not press a button
- Press the wheel up or down
- Press one of the buttons to check the volume
- Repeat the procedure until the desired volume is reached

Changing the volume for one button

- Press and hold the button
- Press the volume control wheel up or down repeatedly until the desired volume is reached

5.10 Application points

Warning

- Only place the SERVOX® digital XL speech aid on uninjured, healthy skin!

Figure (20) Application point at the floor of the mouth

Figure (21) Application point on the cheek

Application points are the places on the throat, floor of the mouth or cheek that allow sound to be transmitted through the tissue when the SERVOX® digital XL speech aid is applied.

Skin folds, scars, beard hair or clothing between the speech aid and the skin can affect the sound quality. The sound should be clean and without noise, such as humming. The optimal application point is the spot that lets the sound through best.

In order to find the best individual application point, the speech aid should be placed in front of a mirror at all application points and tried out.

To try out the sounds, the mouth should be opened slightly and a long stretched vowel such as a long "AAAA" should be spoken.

Tones can be varied by changing the pressure with which the

SERVOX® digital XL is pressed onto the application point or by moving the speech aid slightly up or down from the application point. If possible, practise using the speech aid as part of a speech therapy session.

Note

When the SERVOX® digital XL speech aid is held in the less dominant hand, other objects can be held or used in the dominant hand at the same time.

5.11 Notes on speech formation

- Practise finding your optimal application point again.
- Maintain skin contact even when moving the head or jaw and do not detach the SERVOX® digital XL speech aid from the application point. Loss of contact negatively affects sound quality and intelligibility.
- Imagine whispering very softly. In this way, the breathing noise can be reduced. Practise "pseudo-whispering" – whispering only with the air in your mouth.
- Practise vowels first. Only switch to words when vowels are easy to understand.
- Do not over-articulate. This can result in pharyngeal voice and make you sound unintelligible to your listeners.
- Speak slowly at the beginning and pause appropriately so that your speech sounds as natural as possible.
- Practice speaking only with button A at first. If you feel confident in using button A, add button B as well.
- Practise the use of the tone at the beginning of the speech or at the end of the speech.
- Practise by reading poems or texts aloud
- When speaking, try to maintain your normal facial expressions and gestures.
- Try to stay loose and relaxed, especially in the neck and shoulder area.

6 Troubleshooting

Defect	Possible cause	Solution
SERVOX® digital XL speech aid is not responding	The bottom cover is not tightened, the contact to the batteries is interrupted.	Check the bottom cover. It must be screwed on straight and hand-tight.
	Batteries empty or defective	Replace batteries
SERVOX® digital XL speech aid produces no sound or too weak a sound	The volume is set too low	Increase volume
	The screw cap is loose	Check the screw cap and, if necessary, screw it on straight and hand-tight.
Settings for volume or tones are not saved or cannot be changed despite correct setting methods.	Software error	Contact Orbisana Healthcare GmbH or the local Orbisana Healthcare sales partner and send in the speech aid for maintenance or repair.

7 Accessories

7.1 SERVOX® digital XL screw caps hard tone and soft tone"

Intended purpose: The SERVOX® digital XL screw cap is vibrated by the SERVOX® digital XL speech aid to produce sounds that are transmitted to the mouth via direct contact with throat or cheek tissue and can be used for speaking.

Product description: The SERVOX® digital XL speech aid is delivered with the SERVOX® digital screw cap hard preassembled. The SERVOX® digital screw cap soft produces a more muffled sound in comparison, the SERVOX® digital screw cap hard produces a clearer sound. The top of the SERVOX® digital screw cap is softly coloured to distinguish it.

Replacing the screw cap

- ➔ Before use, check the screw cap for damage. Do not use damaged screw caps.
- ➔ Clean and disinfect the screw cap before first use and before replacement.
- ➔ To replace, unscrew the screw cap from the speech aid and screw on the desired screw cap straight and hand-tight. The ring-shaped, black sealing ring on the (G) thread between the housing of the speech aid and the show cap must not be squeezed.

Figure (22) Screw caps

EN

7.2 SERVOX® digital XL oral tube with adapter

Intended purpose: The SERVOX® digital XL oral tube is used to transmit sound to the oral cavity as an alternative if the SERVOX® digital XL speech aid cannot be applied to one of the application points.

Product description: If it is not possible to place the SERVOX® digital XL speech aid on the skin, e.g. immediately after an operation or during radiation therapy, the speech aid can be used together with the oral tube.

The oral tube and adapter must be cleaned and disinfected before first use and after each use. To use, first put the selected oral tube on the adapter and then push the adapter onto the speech aid's screw cap until it fits securely. The adapter can also be used without an oral tube.

Long tube (2 x)	Short tube (2 x)	Adapter
		
Insert the end of the tube into the corner of the mouth between the upper row of teeth and the cheek (deformable, long tube).	Put the end of the tube a little bit sideways in the corner of the mouth (softer and louder than the long oral tube)	Place the adapter in the corner of the mouth. (louder than both oral tubes)

Figure (23) Oral tubes

Note

The adapter and oral tube must be checked regularly for moisture during use and dried if necessary. The warm and humid environment created by moisture favours the colonisation of bacteria. Moisture can damage the speech aid.

7.3 SERVOX® digital XL strap set

Warning

- Always wear the SERVOX® digital XL speech aid in front of the body in a controlled manner!
- Make sure that SERVOX® digital XL speech aid or strap cannot get caught or jammed!
- Do not leave children unattended with the SERVOX® digital XL speech aid!

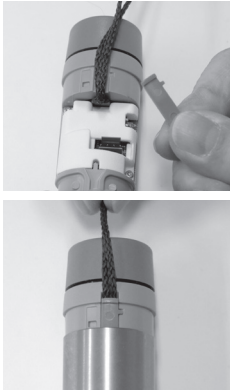
Figure (24) Assembly of the carrying strap

The SERVVOX® digital XL strap set, allows the SERVVOX® digital XL speech aid to be worn around the neck. The carrying strap is inserted underneath the titanium sleeve and secured with the strap protector. One end of the strap protector is bevelled, the other end is U-shaped.

- ➔ Remove titanium sleeve
- ➔ Remove the strap protector from the recess in the top of the device
- ➔ Put the fastener of the carrying strap with the welded end into the recess at the top
- ➔ Place the strap in the notch.
- ➔ Finish by placing the strap protector upside down with the "U-end" over the strap and press it tight
- ➔ Then slide the titanium sleeve back onto the speech aid. If the titanium sleeve does not slide on easily, check the fit of the strap protector and correct it if necessary

If no carrying strap is used, the strap protector should be inserted into the top of the device with the bevelled end so that the opening is closed and no dirt can enter the inside of the housing.

7.4 SERVVOX® digital XL belt pouch

In the SERVVOX® digital XL belt pouch, the SERVVOX® digital XL speech aid can be attached to a belt and carried close to the body. The belt pouch has a separate compartment for battery holders incl. spare batteries and can be adjusted to different belt widths with a hook and loop fastener.

Figure (25) Belt pouch

Store the belt pouch in a dry and dust-free place and dispose of it in the household waste when damaged or worn out

7.5 Batteries, charger and travel adapter (optional)

The batteries can be charged via the battery charger. A travel adapter is included for using the charger with different types of sockets. Always follow the manufacturer's instructions, information and safety instructions for all accessories.

7.6 SERVVOX® digital XL software/SERVVOX® digital XL data cable (can be ordered)

In addition to the SERVVOX® digital XL speech aid, software (CD SERVVOX® digital XL software) and a data cable (SERVVOX® digital XL data cable) can be ordered separately for professional adjustment of the speech melody by medical personnel. Adjustment of the speech melody can be done in a speech therapy session.

8 Cleaning/disinfection

Attention

- Never immerse the SERVVOX® digital XL speech aid in liquids!
- Do not allow any liquids to get inside the SERVVOX® digital XL speech aid!
- Do not clean the SERVVOX® digital XL speech aid with aggressive cleaning agents or aggressive household products!
- Do not use the SERVVOX® digital XL speech aid until it is completely dry!
- The inside of the SERVVOX® digital XL screw cap should not be touched or get wet or come into contact with alcohol.

Note

- Before first use, before prolonged storage and after prolonged non-use, the SERVVOX® digital XL speech aid, screw caps and oral tubes must be cleaned/disinfected.
- Clean or disinfect the SERVVOX® digital XL screw cap only when screwed onto the speech aid.

8.1 Methods used

- For cleaning: Commercial dishwashing detergent
- For disinfection: 94 % alcohol
- Soft and lint-free cloths
- Commercially available cotton swabs
- Access to clear, clean water
- Container for boiling

8.2 SERVVOX® digital XL speech aid and SERVVOX® digital XL screw caps

- ➔ To clean before first use and daily, wipe thoroughly with a soft, lint-free cloth moistened with water and a few drops of commercial dishwashing detergent (not wet or dripping) and dry with a soft lint-free cloth.
- ➔ To disinfect, wipe thoroughly with a soft, lint-free cloth moistened with 94 % alcohol
- ➔ Then dry thoroughly with a soft lint-free cloth or allow to air dry completely.

When cleaning and disinfecting, be sure to also thoroughly wipe the gaps between the screw cap and the speech aid and between the screw cap rim and the membrane, the insert for the carrying strap and around the buttons and volume control to remove any residues of skin grease, skin particles, microorganisms or other dirt. Do not allow any liquid to get inside the SERVVOX® digital XL speech aid.

Battery contacts

- ➔ Monthly check the battery contacts inside the speech aid and on the battery holder for corrosion or dirt.
- ➔ Gently and gently wipe dirty or corroded battery contacts with a cotton swab moistened with 94 % alcohol (not dripping) and allow to dry completely.

8.3 SERVOX® digital XL oral tubes and adapter

Disassemble SERVOX® digital XL oral tubes and SERVOX® digital XL adapter before cleaning and disinfecting. Both must be completely dry before use.

Adapter

- To clean, place in water with a few drops of washing-up liquid once a day and wipe the outside with a lint-free, soft cloth, the inside carefully with a cotton bud and then rinse with clear water and leave to dry completely standing on a paper towel.
- For disinfection, wipe thoroughly once a day after cleaning inside and out with a soft, lint-free cloth moistened with 94 % alcohol. Wipe inside with a cotton swab moistened with 94 % alcohol and then rinse thoroughly with clear water and allow to dry completely while standing on a paper towel.

Oral tube

- To clean, wipe the top end of the tube thoroughly once a day with a soft, lint-free cloth moistened with water and a few drops of commercial dishwashing detergent, then rinse the entire tube thoroughly with warm, clear water and leave to dry completely on a paper towel.
- For disinfection, boil the cleaned oral tube without adapter once a day in a pot for 5 minutes in clear water and let it dry completely standing on a paper towel. For the long oral tube, remove the attachment before boiling. Thoroughly wipe the disinfection attachment with a cotton swab soaked in 94 % alcohol, rinse thoroughly with clean water and allow to dry completely on a paper towel.

8.4 SERVOX® digital XL belt pouch and SERVOX® digital XL strap set

The carrying strap must be removed from the SERVOX® digital XL speech aid must be removed for cleaning. All items must be removed from the belt pouch beforehand.

SERVOX® digital XL belt pouch or SERVOX® digital XL carrying strap should be soaked in warm water with a few drops of dishwashing detergent and washed regularly, at least when dirty. Then rinse the belt pouch or carrying strap thoroughly with clear water and allow to dry completely.

EN

9 Maintenance and repair

With correct use, no regular maintenance is necessary. Repairs may only be carried out by specialist personnel authorised by Orbisana Healthcare GmbH.

Should maintenance or repair be necessary, the SERVOX® digital XL must be cleaned and disinfected and sent together with a description of the fault, a copy of the invoice, the charger and the batteries in the original packaging to the local Orbisana Healthcare sales partner or, if the device was purchased directly from Orbisana Healthcare, to Orbisana Healthcare GmbH.

10 Operation, storage and transport

Store or transport the SERVOX® digital XL and the accessories contained inside it in a clean, dry environment free of dust and protected from light.

Description	Storage/transport	Operation
Temperature in °C	0 °C – 40 °C	+5 °C – 40 °C
Relative humidity	Up to 93 % without condensation	15 % – 93 % without condensation
Air pressure	700 hPa – 1060 hPa	700 hPa – 1060 hPa

Technical specifications subject to change.

11 Period of use and disposal

The SERVOX® digital XL as well as the charger and travel adapter are subject to EC Directive 2012/19/EU (WEEE) and, like rechargeable batteries and batteries, must not be disposed of with normal household waste. For disposal, collection points have been set up at the public waste management authorities (municipalities) to take back old appliances from private households free of charge. Batteries and rechargeable batteries are taken back free of charge by retailers.

Service life	
SERVOX® digital XL speech aid	Can be used by one user for 10 years if the instructions for use are followed.
SERVOX® digital XL screw caps	In case of damage or wear, dispose of with household waste
SERVOX® digital XL oral tubes	In case of damage, discoloration or wear, after infection or after 30 days at the latest, dispose of with household waste.
SERVOX® digital XL adapter for oral tubes	In case of damage, discoloration or wear, dispose of with household waste.
SERVOX® digital XL belt pouch, SERVOX® digital XL strap set	In case of damage or wear, dispose of with household waste
Rechargeable batteries	Observe the manufacturer's instructions. Batteries must be checked and replaced regularly.
Charger Varta LCD Plug, kwmobile universal travel adapter	Observe the manufacturer's instructions. SERVOX® digital XL speech aid, SERVOX® digital XL screw caps, SERVOX® digital XL oral tubes

Due to country-specific requirements, the speech aid is delivered with different accessories.

37731/Version 2023-02-13

12 Technical data

Dimensions	L 128 x Ø 35 mm
Weight	approx. 110 g without rechargeable batteries/batteries
Application parts	SERVOX® digital XL speech aid, SERVOX® digital XL screw caps, SERVOX® digital XL oral tubes
Material	Housing and adapter: BBS plastic, Protective sleeve: Titanium, Screw cap hard: Polymethyl methacrylate (PMMA), Screw cap soft: Thermoplastic elastomers/polypropylene (TPE/PP), Oral tubes: Silicone

Specifications for rechargeable batteries/batteries

Voltage:	1.2 V/1.5 V
Typ. capacity:	750 mAh – 900 mAh
Rechargeable battery type:	AAA NiMH (HR03)
Typ. battery life:	≤ 1000 cycles

Other notes

The talk time depends on the selected battery capacity. The battery life specified by the manufacturer must be observed. The batteries supplied by Orbisana Healthcare GmbH meet the safety requirements of IEC 62133. All replacement batteries must meet this standard.

Due to country-specific requirements, the speech aid is delivered with different accessories.

12.1 Information on electromagnetic compatibility (EMC)

Medical electrical equipment is subject to special precautions regarding EMC and must be installed in accordance with the EMC instructions below. Portable and mobile RF communication devices may affect medical electrical equipment. The use of accessories, transformers and power supplies other than those specified in this manual may result in increased signal emissions or reduced interference resistance of the device or system.

Guidelines and manufacturer's declaration – electromagnetic emissions

The electrical components of the SERVOX® digital XL are intended for operation in an environment as specified below. The user of the SERVOX® digital XL should ensure that it is operated in such an environment.

Interference emissions test	Compliance	Electromagnetic environment – guidance
RF emissions according to CISPR 11	Class B	The SERVOX® digital XL is suitable for use in all settings, including in residential areas and those directly connected to a public supply grid that also supplies buildings used for residential purposes.
Emission of harmonic oscillations as per IEC 61000-3-2	Not applicable	
Transmission of voltage fluctuations/flicker according to IEC 61000-3-3	Not applicable	

Recommended safety distances between portable and mobile RF telecommunication devices and the SERVOX® speech aids

The electrical components of the SERVOX® digital XL are intended for operation in an electromagnetic environment where the RF disturbances are controlled. The user of the speech aids can thereby help avoid electromagnetic interference by maintaining the minimum distance between portable and mobile RF telecommunications equipment (transmitters) and the speech aids, depending on the output power of the communication device, as indicated below.

Nominal power of the transmitter (W)	Safety distance depends on the transmission frequency (m)		
	150 kHz to 80 MHz $d = \{ 6/10 \} \sqrt{\quad}$	80 MHz to 800 MHz $d = \{ 6/10 \} \sqrt{\quad}$	800 MHz to 2.5 GHz $d = \{ 6/10 \} \sqrt{\quad}$
0.01	0.06	0.06	0.06
0.1	0.19	0.19	0.19
1	0.6	0.6	0.6
10	1.90	1.90	1.90
100	6.00	6.00	6.00

For transmitters whose maximum rated power is not listed above, the recommended safety distance can be determined using the equation in the corresponding column. P is the maximum nominal power rating of the transmitter in watts (W) according to the transmitter manufacturer's specifications.

Guidance and manufacturer's declaration – electromagnetic immunity

The electrical components of the SERVOX® digital XL are intended for operation in an environment as specified below. The customer or the user of the SERVOX® digital XL should ensure that it is operated in such an environment.

Immunity testing	IEC 60601 – test level	Compliance level	Electromagnetic environment – guidance
Electrostatic discharge (ESD) according to IEC 1000-4-2	±8 kV Contact discharge ±15 kV Air discharge	±8 kV Contact discharge ±15 kV Air discharge	Floors should be made of wood, concrete or ceramic tile. If the floor is lined with a synthetic material, the relative humidity must be at least 30 %.

Guidance and manufacturer's declaration – electromagnetic immunity

The electrical components of the SERVOX® digital XL are intended for operation in an environment as specified below. The customer or the user of the SERVOX® digital XL should ensure that it is operated in such an environment.

Immunity testing	IEC 60601 – test level	Compliance level	Electromagnetic environment – guidance
Magnetic fields at the supply frequency (50/60 Hz) according to IEC 31000-4-8	30 A/m	30 A/m	Magnetic fields at supply frequency should correspond to typical values for business and hospital environments.
Emitted RF disturbances as per IEC 61000-4-3	10 V/m 80 MHz to 2.5 GHz	10 V/m	Suggested safety distance: $d = \{ 6/10 \} \sqrt{P}$ $d = \{ 6/10 \} \sqrt{P}$ for 80 MHz to 800 MHz $d = \{ 6/10 \} \sqrt{P}$ for 800 MHz to 2.5 GHz P is the nominal power rating of the transmitter in watts (W) according to the transmitter manufacturer's specifications, and d is the recommended safety distance in meters (m).

13 Order information

Individual components of the SERVOX® digital XL, individual spare parts and additional accessories adapted for use with the SERVOX® digital XL can be ordered separately.

EN

Designation	CPN	Order no.
SERVOX® digital XL speech aid complete set	14162696	37575
SERVOX® digital XL Basic – Basic set without charger, adapter and batteries (Australia only)	–	38035

Accessories and spare parts

SERVOX® digital XL screw cap hard tone	14162746	22774
SERVOX® digital XL screw cap soft tone	14162723	23285
SERVOX® digital XL oral tube with adapter	14162700	18253
SERVOX® digital XL strap set	–	17093
SERVOX® digital XL belt pouch	–	16555
SERVOX® digital XL battery holder	–	37574
Rechargeable battery type AAA	–	37561
Charger Varta LCD plug	–	41370
kwmobile universal travel adapter	–	37512
SERVOX® digital XL instruction manual	–	37731
DS SERVOX® digital XL – First Steps for Using (DE/GB)	–	37595
SERVOX® digital XL bottom cover	–	37505
SERVOX® digital XL O-ring	–	14831

Additional accessories can be ordered:

SERVOX® digital XL data cable	14162717	37535
SERVOX® digital XL software	–	38029

Note

The accessories are adapted for use with the SERVOX® digital XL. Non-approved accessories may impair the functions or damage the speech aid.

- Only use the speech aid with the approved accessories!
- Only use approved accessories and spare parts from Orbisana Healthcare!

14 Disclaimer

Repairs or modifications to the SERVOX® digital XL or its components/accessories by persons not authorised by Orbisana Healthcare GmbH are prohibited. In the event of damage, the manufacturer does not assume liability in case of non-compliance with the instruction manual or the safety instructions or manipulation of the product. In the event of non-observance or repairs or modifications to the SERVOX® digital XL or its components/accessories by unauthorised persons will invalidate the warranty.

Only original spare parts and accessories may be used. The warranty is 24 months for parts and material. Excluded are wear parts and consumables. Shipping costs are the responsibility of the owner or warranty claimant. The warranty period begins on the date of delivery.

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