**SERVOCURA®**

unsteril  
non-sterile

**Urin-Beinbeutel 750 ml**

**Leg bag 750 ml**

**S-UB 750**

DE

EN

max. 7 Tage | max. 7 days

Bestell-Nr. **41144**

REF **25107**

PZN **17364350**

**Orbisana Healthcare GmbH**

Biberweg 24-26  
53842 Troisdorf | Germany  
Tel. +49 (0)2241 9322-0  
Fax +49 (0)2241 9322-277  
info@orbisana.de  
www.orbisana.de

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## SERVOCURA® Urin-Beinbeutel 750 ml

### S-UB 750

**unsteril**

Verwendete Symbole ..... 8

## 1 Vorwort

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EN

## 1 Foreword

This instruction manual is intended for patients, relatives and medical staff. The SERVOCURA® leg bag can be used at home and in a clinical setting. Independent use (without support from relatives or nursing staff, for example) is possible. The SERVOCURA® leg bag should be used in line with the descriptions in this instruction manual and in accordance with the recognised rules of medicine.

### Note

Read this instruction manual carefully. It contains important information on safe and proper use of the product. Keep this instruction manual for future reference.

SERVOCURA® urine leg bags are assigned to batches for traceability purposes. The batch number of each SERVOCURA® leg bag is indicated on the instruction manual next to "LOT".

This batch number allows each bag to be uniquely identified. If you have any questions or problems with your SERVOCURA® urine bag, please contact your field representative or Orbisana Healthcare GmbH, referencing this batch number.

### 1.1 Intended purpose

SERVOCURA® bedside drainage bags are connected to urinary sheaths or urinary catheters and collect drained urine.

### Note

The SERVOCURA® urine leg bag is a sterile, disposable medical product and must not be reused, cleaned or reprocessed.

### 1.2 Indication

Urine drainage; treating moderate to severe incontinence.

### 1.3 Contraindication

Known sensitivity to the materials used.

39955/Version 2025-05-05

# 2 Safety instructions

Safe and proper use of your urine bag is important for your health. Ignoring the safety instructions can have significant effects on your health or even cause life-threatening injuries.

**Caution:** The word "Caution" indicates a situation that can result in minor or moderate injury if not avoided.

**Please observe the following safety instructions:**

## Caution

**Risk of infection:** Using a urine bag for longer than intended can cause an infection.

- Replace the bag at least after 7 days.

**Risk of infection:** If the packaging or the product is damaged or contaminated, this may cause an infection.

- Do not use the product if the packaging or the product is damaged!

**Risk of injury:** If the urine bag is attached lower than your bladder, gravity causes the urine to flow down into the bag. If the urine bag is attached higher than your bladder, the urine cannot flow into the bag. This can cause a build-up in your urinary tract and bladder.

- Always attach the bag lower than your bladder so that the urine can flow out!

**Risk of injury:** If the urine bag is full and is not emptied in good time or if the tube is bent, this can cause a build-up in your urinary tract and bladder.

- Empty the urine bag in good time!
- Make sure that you do not bend the tube!

# 3 Description

## Components:

- Urine drainage bag with anti-reflux valve, reverse side with fleece coating
- T-tap drain valve with silicone connector (to connect other urine bags)
- Flap for T-tap drain valve
- 50 cm tube with step connector

**Capacity:** 750 ml

**Material:** PVC

The tube can be shortened, a second connector is included.

## Note

Volumes indicated on the bag are only guidelines!

## 4 Application

Wash the hands thoroughly before and after use.

Use gloves.

A standard skin disinfectant can be used to disinfect the T-tap drain valve or the connection point between the urinary catheter and the bag.

### 4.1 Preparation

- Check the packaging for damage or impurities.
- Check the bag for damage or impurities.
- Close the T-tap drain valve (if open).
- Insert the T-tap drain valve into the flap provided.
- If necessary, shorten the tube using disinfected scissors and attach the connector to the tube.
- Remove the protective cap on the tube connector.
- Disinfect the connection points between the catheter and the bag.

### 4.2 Connecting and securing the bag

Hold the catheter securely and insert the connector of the bag into the funnel connector of the catheter.

Secure the bag to the leg, with the fleece side to the skin, using a suitable fastening system (e.g. SERVOCURA® strap, art. 25226). Ensure that the bag is always below the bladder.

### 4.3 Emptying the bag

Remove the T-tap drain valve from the flap, hold the draining device down and open the T-tap drain valve. Once the bag is empty, close the T-tap drain valve, disinfect it and put it back into the flap.

### 4.4 Removing / replacing the bag

Hold the catheter securely and pull the connector of the bag out of the funnel connector of the catheter. Then disinfect the connection points between the catheter and the bag. The bag should be changed at least every 7 days.

### 4.5 Connecting another bag

For larger quantities of urine, e.g. at night, a bedside bag system can also be connected to a leg bag. To do so, connect the bedside drainage bag to the connection tube of the T-tap drain valve on the leg bag. Then open the T-tap drain valve on the leg bag.

To remove the bedside drainage bag, close the T-tap drain valve on the leg bag and insert it in the flap provided.

## 5 Period of use and disposal

The urine bag must not be used for longer than 7 days. The empty bag can be disposed of with household waste.

## 6 Disclaimer

In the event of damage, the manufacturer does not assume liability in case of non-compliance with the instruction manual or the safety instructions or manipulation of the product.

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