unsteril  
non-sterile

# SERVOCURA®

Urin-Bettbeutel 2000 ml

Bedside drainage bag 2000 ml

S-UB 2000

DE

EN

CE MD

REF 25111

Bestell-Nr. **41146**

**Orbisana Healthcare GmbH**

Biberweg 24-26  
53842 Troisdorf | Germany  
Tel. +49 (0)2241 9322-0  
Fax +49 (0)2241 9322-277  
info@orbisana-hc.de  
www.orbisana-hc.de

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## SERVOCURA® Urin-Bettbeutel 2000 ml S-UB 2000

Verwendete Symbole

8

### 1 Vorwort

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8

## 1 Foreword

This instruction manual is intended for patients, relatives and medical staff. The SERVOCURA® bedside drainage bag can be used at home and in a clinical setting. Independent use (without support from relatives or nursing staff, for example) is possible. The SERVOCURA® bedside drainage bag should be used in line with the descriptions in this instruction manual and in accordance with the recognised rules of medicine.

### Note

Read this instruction manual carefully. It contains important information on safe and proper use of the product. Keep this instruction manual for future reference.

SERVOCURA® urine bags are assigned to batches for traceability purposes. The batch number of each SERVOCURA® bag is indicated on the instruction manual next to "LOT".

This batch number allows each bag to be uniquely identified. If you have any questions or problems with your SERVOCURA® urine bag, please contact your field representative or Orbisana Healthcare GmbH, referencing this batch number.

### 1.1 Intended purpose

SERVOCURA® bedside drainage bags are connected to urinary sheaths or urinary catheters and collect drained urine.

### Note

The SERVOCURA® urine bag is a disposable medical product and must not be reused, cleaned or reprocessed.

### 1.2 Indication

Urine drainage; treating moderate to severe incontinence.

### 1.3 Contraindication

Known sensitivity to the materials used.

## 2 Safety instructions

Safe and proper use of your urine bag is important for your health. Ignoring the safety instructions can have significant effects on your health or even cause life-threatening injuries.

**Caution:** The word “Caution” indicates a situation that can result in minor or moderate injury if not avoided.

**Please observe the following safety instructions:**

### Caution

**Risk of infection:** Using a urine bag for longer than intended can cause an infection.

- Replace the bag at least after 24 hours.

**Risk of infection:** If the packaging or the product is damaged or contaminated, this may cause an infection.

- Do not use the product if the packaging or the product is damaged!

**Risk of injury:** If the urine bag is attached lower than your bladder, gravity causes the urine to flow down into the bag. If the urine bag is attached higher than your bladder, the urine cannot flow into the bag. This can cause a build-up in your urinary tract and bladder.

- Always attach the bag lower than your bladder so that the urine can flow out!

**Risk of injury:** If the urine bag is full and is not emptied in good time or if the tube is bent, this can cause a build-up in your urinary tract and bladder.

- Empty the urine bag in good time!
- Make sure that you do not bend the tube!

## 3 Description

### Components:

- Urine bag with anti-reflux valve
- 90 cm tube with step connector

### Capacity:

2000 ml

### Material:

PVC

### Note

Volumes indicated on the bag are only guidelines!

## 4 Application

Wash the hands thoroughly before and after use.  
Use gloves.

A standard skin disinfectant can be used to disinfect the connection point between the urinary catheter and the bag.

### 4.1 Preparation

- Check the packaging for damage or impurities.
- Check the bag for damage or impurities.
- Remove the protective cap on the tube connector.
- Disinfect the connection points between the catheter and the bag.

### 4.2 Connecting and securing the bag

Hold the catheter securely and insert the connector of the bag into the funnel connector of the catheter.

Secure the bag to the bed using a suitable fastening system (e.g. SERVOCURA® universal holder, art. 25275). Ensure that the bag is always hanging vertically below the bladder of the patient.

### 4.3 Removing / replacing the bag

Hold the catheter securely and pull the connector of the bag out of the funnel connector of the catheter. Then disinfect the connection points between the catheter and the bag. The bag should be changed after 24 hours at the latest.

### 4.4 Connecting another bag

For larger quantities of urine, e.g. at night, a bedside bag system can also be connected to a leg bag. To do so, connect the bedside drainage bag to the connection tube of the T-tap drain valve on the leg bag. Then open the T-tap drain valve on the leg bag.

To remove the bedside drainage bag, close the T-tap drain valve on the leg bag.

## 5 Period of use and disposal

The bag should be changed after 24 hours at the latest. The empty bag can be disposed of with household waste.

## 6 Disclaimer

In the event of damage, the manufacturer does not assume liability in case of non-compliance with the instruction manual or the safety instructions or manipulation of the product.

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