steril
sterile

SERVOCURA®

Urin-Bettbeutel 2.000 ml

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Bedside drainage bag 2.000 ml

with T-tap drain valve

DE

EN

S-UB 2000 T-Tap

max. 7 Tage | max. 7 days

Orbisana Healthcare GmbH

Biberweg 24-26

53842 Troisdorf | Germany

Tel. +49 (0)2241 9322-0

Fax +49 (0)2241 9322-277

info@orbisana.de

www.orbisana.de

REF 25108

CE 0197 **MD**

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SERVOCURA® **steril**

Urin-Bettbeutel 2.000 ml mit T-Tap Ventil

S-UB 2000 T-Tap

Verwendete Symbole 8

1 Vorwort

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INSTRUCTION FOR USE

SERVOCURA® **sterile** **bedside drainage bag 2.000 ml** with T-tap drain valve **S-UB 2000s T-tap**

Symbols used 8

1 Foreword

EN

This instruction manual is intended for patients, relatives and medical staff. The SERVOCURA® bedside drainage bag can be used at home and in a clinical setting. Independent use (without support from relatives or nursing staff, for example) is possible. The SERVOCURA® bedside drainage bag should be used in line with the descriptions in this instruction manual and in accordance with the recognised rules of medicine.

Note

Read this instruction manual carefully. It contains important information on safe and proper use of the product. Keep this instruction manual for future reference.

SERVOCURA® bedside drainage bags are assigned to batches for traceability purposes. The batch number of each SERVOCURA® bedside drainage bag is indicated on the instruction manual next to "LOT". This batch number allows each bag to be uniquely identified. If you have any questions or problems with your SERVOCURA® urine bag, please contact your field representative or Orbisana Healthcare GmbH, referencing this batch number.

1.1 Intended purpose

SERVOCURA® bedside drainage bags are connected to urinary sheaths or urinary catheters and collect drained urine.

Note

The SERVOCURA® bedside drainage bag is a sterile, disposable medical product and must not be reused, cleaned or reprocessed.

1.2 Indication

Urine drainage; treating moderate to severe incontinence.

1.3 Contraindication

Known sensitivity to the materials used.

1.4 Side effects

There are no known side effects provided the patient uses the product properly and observes the safety instructions.

2 Safety instructions

Safe and proper use of your urine bag is important for your health. Ignoring the safety instructions can have significant effects on your health or even cause life-threatening injuries.

Attention: The word “Attention” indicates a situation that can cause damage to your product if not avoided.

Caution: The word “Caution” indicates a situation that can result in minor or moderate injury if not avoided.

Please observe the following safety instructions:

Caution

Risk of infection: Using a urine bag for longer than intended can cause an infection.

- Replace the bag at least after 7 days.

Risk of infection: If the packaging or the product is damaged or contaminated, the product is no longer sterile. This may cause an infection.

- Do not use the product if the packaging or the product is damaged!

Risk of injury: If the urine bag is attached lower than your bladder, gravity causes the urine to flow down into the bag. If the urine bag is attached higher than your bladder, the urine cannot flow into the bag. This can cause a build-up in your urinary tract and bladder.

- Always attach the bag lower than your bladder so that the urine can flow out!

Risk of injury: If the urine bag is full and is not emptied in good time or if the tube is bent, this can cause a build-up in your urinary tract and bladder.

- Empty the urine bag in good time!
- Make sure that you do not bend the tube!

Attention

The product is sterilised with ethylene oxide. Resterilisation may damage the product!

- Do not resterilise the product!

3 Description

Components:

- Urine drainage bag with anti-reflux valve
- T-tap drain valve
- 90 cm tube with step connector

Capacity:

2.000 ml

Material:

PVC

Note

Volumes indicated on the bag are only guidelines!

4 Application

Wash the hands thoroughly before and after use.

A standard skin disinfectant can be used to disinfect the T-tap drain valve or the connection point between the urinary catheter and the bag.

4.1 Preparation

- Check the packaging for damage or impurities.
- Remove the bag from the sterile packaging. Make sure that you do not contaminate the connector.
- Check the bag for damage or impurities.
- Close the T-tap drain valve (if open).
- Insert the T-tap drain valve into the flap provided.
- If necessary, shorten the tube using disinfected scissors and attach the connector to the tube.
- Remove the protective cap on the tube connector.
- Disinfect the connection points between the catheter and the bag.

4.2 Connecting and securing the bag

To connect the bag to the catheter, hold the catheter securely and insert the connector of the bag into the funnel connector of the catheter.

Secure the bag to the leg, with the fleece side to the skin, using a suitable fastening system (e.g. SERVOCURA® strap, art. 25226). Ensure that the bag is always below the bladder.

4.3 Emptying the bag

To empty the bag, remove the T-tap drain valve from the flap, hold the draining device down and open the T-tap drain valve. Once the bag is empty, close the T-tap drain valve, disinfect it and put it back into the flap.

4.4 Removing / replacing the bag

To remove the bag, hold the catheter securely and pull the connector of the bag out of the funnel connector of the catheter. Then disinfect the connection points between the catheter and the bag.

The bag should be changed at least every 7 days.

4.5 Connecting another bag

For larger quantities of urine, e.g. at night, a bedside bag system can also be connected to a leg bag. To do so, connect the bedside drainage bag to the connection tube of the T-tap drain valve on the leg bag. Then open the T-tap drain valve on the leg bag.

To remove the bedside drainage bag, close the T-tap drain valve on the leg bag and insert it in the flap provided.

5 Period of use and disposal

The urine bag must not be used for longer than 7 days. The empty bag can be disposed of with household waste.

6 Disclaimer

In the event of damage, the manufacturer does not assume liability in case of non-compliance with the instruction manual or the safety instructions or manipulation of the product.

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