steril
sterile

SERVOCURA®

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INSTRUCTIONS FOR USE

SERVOCURA® **sterile** bedside bag 2.000 ml bedside bag closed system S-UDS 2000s

Symbols used	8
--------------------	---

EN

1 Foreword

This instruction manual is intended for patients, relatives and medical staff. The SERVOCURA® bedside bag closed system can be used at home and in a clinical setting. Independent use (without support from relatives or nursing staff, for example) is possible. The SERVOCURA® bedside bag closed system should be used in line with the descriptions in this instruction manual and in accordance with the recognised rules of medicine.

Note

Read this instruction manual carefully. It contains important information on safe and proper use of the product. Keep this instruction manual for future reference

SERVOCURA® urine bags are assigned to batches for traceability purposes. The batch number of each SERVOCURA® bag is indicated on the instruction manual next to "LOT".

This batch number allows each bag to be uniquely identified. If you have any questions or problems with your SERVOCURA® urine bag, please contact your field representative or Orbisana Healthcare GmbH, referencing this batch number.

1.1 Intended purpose

The SERVOCURA® urinary drainage system is connected to urinary sheaths or urinary catheters and collects drained urine.

Note

The SERVOCURA® bedside bag closed system is a sterile, disposable medical product and must not be reused, cleaned or reprocessed.

1.2 Indication

Urine drainage; treating moderate to severe incontinence.

1.3 Contraindication

Known sensitivity to the materials used.

39962/Version 2025-04-14

2 Safety instructions

Safe and proper use of your urine bag is important for your health. Ignoring the safety instructions can have significant effects on your health or even cause life-threatening injuries.

Attention: The word "Attention" indicates a situation that can cause damage to your product if not avoided.

Caution: The word "Caution" indicates a situation that can result in minor or moderate injury if not avoided.

Please observe the following safety instructions:

Caution

Risk of infection: Using a urine bag for longer than intended can cause an infection.

- Replace the bag at least after 14 days.

Risk of infection: If the packaging or the product is damaged or contaminated, the product is no longer sterile. This may cause an infection.

- Do not use the product if the packaging or the product is damaged!

Risk of injury: If the urine bag is attached lower than your bladder, gravity causes the urine to flow down into the bag. If the urine bag is attached higher than your bladder, the urine cannot flow into the bag. This can cause a build-up in your urinary tract and bladder.

- Always attach the bag lower than your bladder so that the urine can flow out!

Risk of injury: If the urine bag is full and is not emptied in good time or if the tube is bent, this can cause a build-up in your urinary tract and bladder.

- Empty the urine bag in good time!
- Make sure that you do not bend the tube!

Attention

The product is sterilised with ethylene oxide. Resterilisation may damage the product!

- Do not resterilise the product!

3 Description

Components:

- Urine drainage bag with a 3-part ventilated anti-reflux drop chamber
- T-tap drain valve
- Flap for T-tap drain valve
- Integrated holder system comprising a bed hook and a strap
- 120 cm tube with step connector with a needle-free sampling port
- Fixing clamp and sliding clamp to build up urine in the tube

Capacity:

2.000 ml

Material:

PVC

Note

Volumes indicated on the bag are only guidelines!

4 Application

Wash the hands thoroughly before and after use.

A standard skin disinfectant can be used to disinfect the T-tap drain valve or the connection point between the urinary catheter and the bag.

4.1 Preparation

- Check the packaging for damage or impurities.
- Remove the bag from the sterile packaging. Make sure that you do not contaminate the connector.
- Check the bag for damage or impurities.
- Close the T-tap drain valve (if open).
- Insert the T-tap drain valve into the flap provided.
- Remove the protective cap on the tube connector.
- Disinfect the connection points between the catheter and the bag.

4.2 Connecting and securing the bag

Hold the catheter securely and insert the connector of the bag into the funnel connector of the catheter.

Secure the bag with the bed hook and the strap. Secure the tube with the fixing clamp. Ensure that the bag is always hanging vertically below the bladder of the patient.

4.3 Emptying the bag

Remove the T-tap drain valve from the flap, hold the draining device down and open the T-tap drain valve. Once the bag is empty, close the T-tap drain valve, disinfect it and put it back into the flap.

4.4 Removing / replacing the bag

Hold the catheter securely and pull the connector of the bag out of the funnel connector of the catheter. Then disinfect the connection points between the catheter and the bag. The bag should be changed at least every 14 days.

4.5 Collecting fresh urine

Use the sliding clamp to seal the tube close to the sampling port on the connector. Then remove the build-up of urine at the sampling port using a needle-free Luer syringe (not included). After, open the sliding clamp on the tube.

5 Period of use and disposal

The urine bag must not be used for longer than 14 days. The empty bag can be disposed of with household waste.

6 Disclaimer

In the event of damage, the manufacturer does not assume liability in case of non-compliance with the instruction manual or the safety instructions or manipulation of the product.

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