SERVOX[®] Rota Tube

basic | voice | cuff | cuff voice

Orbisana Healthcare GmbH

Biberweg 24-26
53842 Troisdorf | Germany
Tel. +49 (0)2241 9322-0
Fax +49 (0)2241 9322-271
info@orbisana-hc.de
www.orbisana-hc.de

CE 0197

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SERVOX® Rota Tube

basic | voice | cuff | cuff voice

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Inhalt

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EN

Contents

| | | |
|-----------|---|-----------|
| 1 | Foreword | 18 |
| 1.1 | Intended purpose | 18 |
| 1.2 | Indication | 18 |
| 1.3 | Contraindication | 18 |
| 1.4 | Side effects | 19 |
| 2 | Safety instructions | 19 |
| 3 | Scope of delivery | 22 |
| 3.1 | Cannula card | 22 |
| 4 | Description | 22 |
| 4.1 | SERVOX® Rota Tube voice/SERVOX® Rota Tube cuff voice (cannulas with fenestration) | 22 |
| 4.2 | SERVOX® Rota Tube cuff/SERVOX® Rota Tube cuff voice (cannulas with cuff) | 23 |
| 4.3 | Tracheal cannula connections | 23 |
| 5 | Tracheal cannula accessories | 23 |
| 5.1 | Remove wedge SERVOX® RW15 | 23 |
| 6 | Application | 24 |
| 6.1 | Preparing and checking tracheal cannulas without cuff | 24 |
| 6.2 | Preparing and checking tracheal cannulas with cuff | 24 |
| 6.2.1 | Checking the cuff and safety valve | 24 |
| 6.3 | Using the 15 mm standard connector | 25 |
| 6.4 | Inserting the outer cannula without cuff | 25 |
| 6.5 | Inserting the outer cannula with cuff | 25 |
| 6.6 | Changing the cannula without cuff | 26 |
| 6.7 | Changing the cannula with cuff | 26 |
| 7 | Cleaning and disinfection of tracheal cannulas and accessories | 26 |
| 7.1 | Safety instructions for cleaning and disinfection | 27 |
| 7.2 | Required aids and accessories | 27 |
| 7.3 | Cleaning instructions | 27 |
| 7.4 | Disinfection instructions | 28 |
| 8 | Storage and transport | 29 |
| 9 | Service life and disposal | 29 |
| 10 | Technical data | 29 |
| 10.1 | Parameters of the SERVOX® Rota Tube | 29 |
| 11 | Order information, replacement parts and accessories | 30 |
| 12 | Disclaimer | 30 |

40860/Version 2022-10-24

1 Foreword

Dear customer,

We would like to thank you for your trust and the purchase of our quality product.

This instruction for use is intended for patients, relatives and medical staff. The tracheal cannula is suitable for all age groups for use at home and in a clinical setting, depending on the selected size. The patient can use the product independently (without the support of relatives or carers), but must be given instructions on its use by their doctor or by medical staff beforehand. Tracheal cannulas must be prescribed by a doctor. The doctor also determines the size, shape and type of the prescribed tracheal cannula. The tracheal cannula must be used in line with the descriptions in this instruction manual and in accordance with the recognised rules of medicine.

Tracheal cannulas and their accessories are assigned a batch number for traceability purposes. The batch number is indicated on the packaging next to "LOT". If you have any questions or problems with your product, please contact your field representative or Orbisana Healthcare GmbH, referencing this batch number.

Note

These instructions for use contain important information for safe and proper use. Keep this instructions for use for future reference.

1.1 Intended purpose

All SERVVOX® Rota Tube versions are used in an artificial opening in the trachea (tracheostoma) in order to keep the tracheostoma open and to stabilise it. All SERVVOX® Rota Tube versions are suitable for use in ventilated patients.

The SERVVOX® Rota Tube voice and SERVVOX® Rota Tube cuff voice can also be used for speaking in combination with a speech valve by patients with a voice option.

The SERVVOX® Rota Tube cuff and SERVVOX® Rota Tube cuff voice can also be used for aspiration management.

Note

The tracheal cannula is intended for reuse on a single patient and must not be reused on any other person. There is a risk of infection.

1.2 Indication

- SERVVOX® Rota Tube basic and SERVVOX® Rota Tube cuff: Use in spontaneously breathing and ventilated tracheotomised patients.
- SERVVOX® Rota Tube voice and SERVVOX® Rota Tube cuff voice: Use in spontaneously breathing and ventilated tracheotomised patients with a voice option.

1.3 Contraindication

- Tracheostoma is too small or in an unsuitable position
- Abnormal anatomy or pathology of the upper respiratory tracts
- Unusually deep trachea (e.g. in obese patients)
- Known allergies to polyvinyl chloride (PVC).

- No use as pediatric cannula with a required inner diameter < 6.0mm
- In laryngectomised patients, the tracheal cannulas must not be used with a speech valve for tracheotomised patients.

1.4 Side effects

- Contamination of the surrounding tissue due to colonisation of the cannulas with e.g. yeast fungus (candida), bacteria, etc.
- Formation of haematomas (particularly during first use)
- Formation of granulation tissue or blocking of the fenestration in fenestrated cannulas with heavy or stringy secretions
- When using fenestrated tracheal cannulas for ventilation, the fenestration may cause leaks in the ventilation system
- Risk of complications with tracheal cannulas with cuff e.g. pressure necrosis due to excessive cuff pressure or aspiration or loss of air during ventilation with a low cuff pressure

EN

Note

Wearing a tracheal cannula may encourage an overprotective posture. It may also cause or exacerbate swallowing difficulties. If you have any questions or if complications occur, please contact your doctor.

2 Safety instructions

Safe and proper use of your tracheal cannula is important for your health. Ignoring these safety instructions can have significant effects on your health or even cause life-threatening injuries.

Attention: The word "Attention" indicates a situation that can cause damage to your product if not avoided.

In the following, warnings on the safe use of the product are indicated with the  symbol.

Danger: The word "Danger" indicates a likely situation that can result in grave injury or even death if not avoided.

Warning: The word "Warning" indicates a situation that can result in grave injury or even death if not avoided.

Caution: The word "Caution" indicates a situation that can result in minor or moderate injury if not avoided.

Please observe the following safety instructions:

Danger

Risk of suffocation: If your tracheal cannula is blocked, e.g. by secretion or if you use aids such as artificial noses or speech valves and they are blocked by secretions, for example, you may become breathless or suffocate.

- Check the cannula regularly! Change it if necessary!
- Regularly suck secretion out of your cannula!
- Regularly check the attached aids you have put on! Change them if necessary!

Risk of suffocation: It is not possible to exhale through a speech valve. Laryngectomised patients without a voice prosthesis/shunt valve cannot breathe when

Danger

the tracheostoma is blocked. They can get into respiratory distress or suffocate, injure themselves or die.

- For laryngectomised patients, do not use speech valves intended for tracheostomised patients!
- Only use speech valves on awake, responsive patients!
- Never block the speech valve! Remove blocked or clogged speech valves immediately!
- Never block the tracheostoma! Do not insert any objects into the cannula!

Risk of aspiration: If you insert a speech valve directly into your tracheostoma, you may aspirate it. You may injure yourself, become short of breath, suffocate, or die.

- Only place a speech valve on the 15 mm standard connector!

Risk of injury/extubation/suffocation: When using a ventilator the ventilation tube may detach from the cannula or exert too much tension on the cannula. The tracheal cannula dislodge or be pulled out of the tracheostoma. You may injure yourself, become short of breath or suffocate.

- Always secure the tracheal cannula with a cannula strap!
- Secure the connection between the tracheal cannula and the ventilation tube!
- Ensure that the strain on the breathing tube is lessened when ventilating!

Risk of injury with tracheal cannulas with cuff: If you put a speech valve on a blocked tracheal cannula, you cannot exhale. You may go into respiratory distress, suffocate or die.

- In case of narrowed airways, only place speech valves on fenestrated tracheal cannulae!
- Unblock tracheal tubes before putting on a speech valve!

Risk of injury with tracheal cannulas with cuff: During ventilation, oxygen can diffuse into the cuff. The cuff pressure can rise or drop.

- Regularly check the cuff pressure.

Warning

Risk of injury with tracheal cannulas with cuff: Excessive cuff pressure can cause nerve damage or damage to the trachea.

- Ensure that the cuff pressure does not exceed 25 mbar!
- Regularly check the cuff pressure (every 2-3 hours).

Risk of injury: When using local painkillers/anaesthetics the feeling for the fit of the tracheal cannula or the cuff may be impaired.

- Regularly check cannula fit and cuff pressure!

Risk of aspiration with tracheal cannulas with cuff: Secretions can accumulate above a blocked cuff.

- Always suction before unblocking the cannula!

Risk of injury/aspiration: If you insert a cleaning brush or similar objects into your tracheostoma, you may aspirate them.

- Do not insert any objects into your tracheostoma!
- Only ever clean your tracheal cannula outside of your body!

Risk of inflammation/irritation: Residues of cleaning agents and disinfectants on the tracheostomy or the tracheal cannula can lead to irritation or inflammation.

- Always rinse the tracheal cannula thoroughly after cleaning and disinfection!

Risk of aspiration: Immediately after a meal, food residues may remain in the throat and mouth and can be aspirated.

- Do not change/insert tracheal cannulae until approx. 30 min after a meal!

Caution

Risk of injury: If a tracheal cannula shows signs of damage such as changes, discolouration or damage to the surfaces or dents or wrinkles in the cuff, you may injure yourself.

- Do not use a damaged or altered tracheal cannula!
- Replace the tracheal cannula immediately even if it is slightly damaged!
- Do not modify your tracheal cannula yourself! All adjustments, such as sieving or shortening, must be prescribed by the attending physician and may only be carried out by the manufacturer!

Risk of injury: When attaching or removing aids such as artificial noses or speech valves of or to an inserted tracheal cannula, with too much pressure or traction on the tracheostomy tube can injure your trachea.

- Hold the neck flange when attaching or removing aids!
- Adjust pressure or tension carefully!
- If necessary, use a remove wedge!

Risk of respiratory distress: If a tracheal cannula is inserted, the airway resistance increases. Aids such as speech valves or artificial noses further increase the breathing resistance.

- In case of sudden shortness of breath, remove all aids immediately!
- Use suction if necessary. If symptoms persist, consult a doctor immediately!

Risk of respiratory distress: When using a speech valve: If the valve cap does not open or if, for example, the speech valve or the valve cap is blocked or stuck due to secretion, you may experience respiratory distress.

- Make sure that the valve cap suspension is always up and that the valve cap can move freely!
- Remove the speech valve immediately in case of breathing difficulties!
- Do not bend over for a longer period while wearing a speech valve!
- Do not wear the speech valve while lying down! Do not sleep with the speech valve on!
- Clean the speech valve at least 1-2 times a day!

Risk of respiratory distress/disconnection: Lubricant residues within the inside the cannula can make breathing difficult. Lubricant residues on the 15 mm standard connector can cause the disconnection of ventilators or aids connected to it.

- Make sure to carefully remove any residual lubricant before inserting the cannula.

Risk of allergy: Lubricants for tracheal cannulae may cause allergic reactions.

- If you have an allergic reaction, do not use the lubricant!
- If you have known allergies, check the ingredients of the lubricant!

MRI safety information

Medical devices with an MR safe symbol
are safe for MRI.

Our tracheostomy tubes can be used during an X-ray imaging.

Note

Please inform us or the responsible authorities if you experience a life-threatening illness or injury or permanent physical damage as a result of using our product, or require hospital treatment or have to extend hospital treatment due to using our product.

3 Scope of delivery

- SERVOX® Rota Tube basic:
1 tracheal cannula with flexible 15 mm standard connector, 1 tube holder, 1 obturator, 1 remove wedge RW 15, 1 cannula card, 2 labels and 1 instruction for use.
- SERVOX® Rota Tube voice:
1 fenestrated tracheal cannula with flexible 15 mm standard connector, 1 tube holder, 1 obturator, 1 remove wedge RW 15, 1 cannula card, 2 labels and 1 instruction for use.
- SERVOX® Rota Tube cuff:
1 tracheal cannula with flexible 15 mm standard connector and low pressure cuff, 1 tube holder, 1 obturator, 1 remove wedge RW 15, 1 cannula card, 2 labels and 1 instruction for use.
- SERVOX® Rota Tube cuff voice:
1 fenestrated tracheal cannula with flexible 15 mm standard connector and low pressure cuff, 1 tube holder, 1 obturator, 1 remove wedge RW 15, 1 cannula card, 2 labels and 1 instruction for use.

3.1 Cannula card

All cannulas come with a cannula card. The patient's data and the attending physician can be entered. One of the enclosed labels can be attached to the back of the labels with the information and batch number of the cannula.

Always carry the cannula passport with you and keep it handy so that the necessary information is available on request or in the event of an accident.

4 Description

The tracheal cannulas are made of latex-free and DEHP-free PVC. The cannula has an angle of 100°. The cannula is inserted into the windpipe. The neck flange is located at the top end of the tube of the cannula with 2 loops on the edge of the flange to secure a holder. The cannula size, length and outer dimensions are specified on the neck flange to identify the cannula. The labelling on the neck flange is green for non-fenestrated tracheal cannulas and blue for fenestrated cannulas. The enclosed insertion aid (obturator) facilitates the insertion of the cannula and can be used to stabilise the cannula during storage or transport. The obturator must be removed from the cannula immediately after insertion. All SERVOX® Rota Tube variants are sterile on delivery and must not be resterilised.

4.1 SERVOX® Rota Tube voice/SERVOX® Rota Tube cuff voice (cannulas with fenestration)

For patients with a voice option, the tracheal cannula is fenestrated at the top of the bend. Deliberately and briefly sealing the tracheostoma can guide the exhaled air through this fenestration over the larynx through the vocal folds and can be used to speak.

4.2 SERVOX® Rota Tube cuff/SERVOX® Rota Tube cuff voice (cannulas with cuff)

The tube of the outer cannula is surrounded by an inflatable (blockable) low-pressure cuff. A pilot balloon (cuff pilot) is connected to the cuff with a supply tube. The valve on the cuff pilot has a connection to attach a cuff pressure gauge or a syringe with a Luer connection to fill or release air. To block (inflate) the cuff, it is filled with air through this valve. To unblock the cuff, the air is released. The cuff pressure is also checked using the cuff pilot. A cuff pressure monitor must always be used to block the cuff and check the cuff pressure.

A blocked cuff prevents or limits secretions from the upper airways from entering the lungs (aspiration) and air flowing outside the cannula or between the tracheal wall and the cannula when breathing. Blockable tracheal cannulas must therefore be unblocked if a speech valve or closure cap is attached.

The doctor determines the cuff pressure. The cuff pressure must normally not exceed 25 mbar. Since movements or coughing can change the cuff pressure, it must be regularly checked, around every 2-3 hours, or more frequently as needed.

Tracheal cannulas with cuff are usually used in patients who cannot safely swallow (risk of aspiration) or who are invasively ventilated.

For patients with a voice option, the outer cannula of the SERVOX® Rota Tube cuff voice is fenestrated above the cuff at the top of the bend and the inner cannula at the corresponding point. While the tracheal cannula is being used for speaking, e.g. during the day, the cuff must be unblocked.

EN

Note

Your tracheal cannula can be subsequently fenestrated or shortened for the individual patient by the manufacturer or professionals authorised by the manufacturer in consultation with the attending physician. For this purpose, contact your attending physician.

4.3 Tracheal cannula connections

The 15 mm standard connector on the upper end of the inner cannula can be used to attach medical aids, such as closure caps, speech valves or artificial noses or connect a ventilator.

5 Tracheal cannula accessories

5.1 Remove wedge SERVOX® RW15

The remove wedge facilitates the removal of aids, such as speech valves or artificial noses from the 15 mm standard connector. To remove the attached aid, carefully place the remove wedge behind the aid and slowly slide it forwards on the standard connector with a slight lever motion.

The RW15 remove wedge may only be used clean and undamaged and can be cleaned and disinfected together with the tracheal cannula.

6 Application

Warning

- Due to the risk of aspiration, wait at least 30 minutes after eating before inserting a new tracheal cannula.
- Suction any secretions before changing the cannula.
- Always unblock the cuff before attaching a closure cap or a speech valve!
- Always unblock the cuff before changing the position of the tracheal cannula or removing the tracheal cannula.
- Do not bend or pull on the cuff pilot and the supply tube of the cuff!
- Regularly check the cuff pressure! The cuff pressure must not exceed 25 mbar.

Note

- Wash and disinfect your hands before touching the tracheostoma or the tracheal cannulas.
- In order to make it easier to use the inserted tracheal cannula later on, practice releasing and attaching various tools before inserting the cannula.
- Relatives or medical staff who take care of the patient should wear gloves and a face mask and document all measures taken.

6.1 Preparing and checking tracheal cannulas without cuff

- Check that the sterile packaging is not damaged and that all parts are included. Do not use tracheal cannulas if the packaging is damaged!
- Remove the cannula from the packaging and check for damage.
- Check that all parts are included and that the cannula is the correct size. (information on the neck flange). Do not use damaged cannulas, accessories or the wrong size of cannula!
- Check whether the obturator is easy to remove.
- Ensure that the outer cannula is lubricated: To do so, apply a suitable lubricant to the cannula or moisten the cannula with water. Carefully remove any residual lubricant on the holder for the inner cannula, inside the cannula or on the connector.
- Place the tracheal cannula within reach on a clean surface.

6.2 Preparing and checking tracheal cannulas with cuff

- Check that the sterile packaging is not damaged and that all parts are included. Do not use tracheal cannulas if the packaging is damaged!
- Remove the cannula from the packaging and check for damage.
- Check that all parts are included and that the cannula is the correct size. (information on the neck flange). Do not use damaged cannulas, accessories or the wrong size of cannula!
- Check whether the obturator is easy to remove.

6.2.1 Checking the cuff and safety valve

The cuff and cuff pilot must be checked with a cuff pressure monitor to ensure they are sealed and intact every time before they are used. Cannulas with a leaky cuff must not be used. Signs of the cuff leaking also include:

- External damage to the cuff (holes, tears etc.).
- Noticeable hissing due to air escaping from the cuff.
- Cuff cannot be completely filled.
- Water in the supply tube to the cannula, in the cuff or in the control balloon (after cleaning and disinfection!).

Then:

- Ensure that the cannula is lubricated: To do so, apply a suitable lubricant to the cannula or moisten the cannula with water. Carefully remove any residual lubricant inside the cannula or on the connector.
- Place the cannula separately within reach on a clean surface.

6.3 Using the 15 mm standard connector

The 15 mm standard connector is used to attach accessories such as artificial noses, speech valves or similar aids. They should be used after the tracheal cannulas are inserted. Take care to avoid injuring the trachea.

- To attach an aid, hold the neck flange securely and carefully push the aid onto the 15 mm standard connector. The aid must sit securely but still be easily removable! If it is not in the correct position, it may fall out. If the aid is attached after the tracheal cannula is inserted, adjust the pressure on the tracheal cannula accordingly.
- To remove the aid, hold the neck flange securely and carefully pull it off, using a remove wedge if necessary.

6.4 Inserting the tracheal cannula without cuff

- Clean the tracheostoma.
- Put the patient in the supine position with the upper body elevated and the head slightly tilted back.
- Insert the cannula, as practiced, into the tracheostoma. This may cause the urge to cough.
- Immediately remove the obturator!
- Check the cannula is positioned correctly. The tracheal cannula should be in the tracheostoma with as little tension as possible.
- Secure the tracheal cannula: Secure the tube holder in the loops in the neck flange (hooks outwards) and pull tightly in order to prevent the tracheal cannula from slipping out the trachea. Adjust and check the fit. The band must be tight enough to ensure it does not accidentally come loose but not so tight that it constricts the throat! Place a finger between the throat and the tube holder to check.
- Check the position of the cannula again and wait a few minutes in case any complications such as breathing problems, the urge to cough, vomiting etc. occur. The throat may briefly swell up when changing the cannula.

6.5 Inserting the tracheal cannula with cuff

- Clean the tracheostoma.
- Put the patient in the supine position with the upper body elevated and the head slightly tilted back.
- Insert the cannula with the completely unblocked cuff, as practiced, into the tracheostoma. This may cause the urge to cough.
- Immediately remove the obturator!
- Check the cannula is positioned correctly. The tracheal cannula should be in the tracheostoma with as little tension as possible.
- Block the cuff until the trachea is adequately sealed. Check the cuff pressure using the cuff pressure gauge. It must not exceed 25 mbar. Once the cuff has been blocked, the position of the cannula must not be changed!

- Secure the tracheal cannula: Secure the tube holder in the loops in the neck flange (hooks outwards) and pull tightly in order to prevent the tracheal cannula from slipping out the trachea. Adjust and check the fit. The band must be tight enough to ensure it does not accidentally come loose but not so tight that it constricts the throat! Place a finger between the throat and the tube holder to check.
- Check the position of the cannula again and wait a few minutes in case any complications such as breathing problems, the urge to cough, vomiting etc. occur. The throat may briefly swell up when changing the cannula.
- Regularly check the cuff pressure (every 2-3 hours).

6.6 Changing the cannula without cuff

The cannula should be changed and cleaned at an interval determined by the doctor, however no later than after 7 days (or more frequently as needed). Secretions should be suctioned before changing the cannula.

- When changing the cannula, always prepare and have a spare cannula ready.
- Open the tube holder and slowly and carefully remove the outer cannula from the tracheostoma. This may cause an urge to cough.
- Clean the tracheostoma and allow to air dry.
- Insert the new tracheal cannula.
- Clean and disinfect the used tracheal cannula immediately.

6.7 Changing the cannula with cuff

The cannula should be changed and cleaned at an interval determined by the doctor, however no later than after 7 days (or more frequently as needed). Secretions must be suctioned before changing the cannula.

- When changing the cannula, always prepare and have a spare cannula ready.
- Suction secretions above the cuff.
- Completely unblock the cuff.
- Open the tube holder and slowly and carefully remove the outer cannula from the tracheostoma. This may cause an urge to cough.
- Clean the tracheostoma and allow to air dry.
- Insert the new tracheal cannula.
- Clean and disinfect the used tracheal cannula immediately.

7 Cleaning and disinfection of tracheal cannulas and accessories

Note

It is often not possible to clean and disinfect individual tracheal cannulas for reuse on a single patient in a clinical setting or this may contradict internal hospital regulations. Therefore, it is recommended in a clinical setting to use the tracheal cannula SERVOX® Rota Tube as a single-use product and to dispose of it after use. In the domestic environment, the cannula can be cleaned and disinfected up to 8 times as described here.

7.1 Safety instructions for cleaning

Warning

- Only ever clean and disinfect your tracheal cannula outside of your body!
- Do not insert any objects into your tracheostoma!
- Always rinse the tracheal cannula thoroughly after cleaning and disinfection!

Attention

- Only ever use suitable and recommended cleaning and disinfection agents to clean and disinfect your tracheal cannula and the accessories! Do not use any aggressive household cleaners or high-proof alcohol!
- Only use an appropriate cleaning brush to clean your tracheal cannula! Do not use any sharp objects such as scrapers, knives or fingernails. These may damage your tracheal cannula. Clean the tracheal cannula and the accessories before disinfection!
- Do not brush the cuff in order to avoid damage!
- The cuff pilot must not be immersed in a cleaning solution!
- Do not resterilise the tracheal cannula!

EN

7.2 Required aids and accessories

- Cannula cleaning brush in the right size. The cleaning brush should be flexible, with soft bristles and a rounded tip in order to prevent damage to the cannula.
- Suitable plastic cleaning tub with a sieve insert (e.g. SERVIX® cleaning tub with sieve insert)
- (Cannula) cleaning agent for alkali-stable instruments and devices in all medical areas and laboratories (biodegradable surfactants: non-ionic surfactants < 5%, anionic surfactants 5-15%, phosphates > 30%).
- Cannula disinfectant for plastic tracheal cannulas
- Gloves
- Access to warm, clean, and running water

7.3 Cleaning instructions

- Wash hands.
- Remove any attached accessories and also clean or dispose of them (observing the respective instruction manuals).
- Inflate the cuff before cleaning.
- To remove larger impurities, carefully flush the tracheal cannulas and the accessories with lukewarm running water.
- Remove firmly attached secretions with the cannula cleaning brush. Bend the brush into the same shape as the cannula before use.
- Always insert the cleaning brush from the tip of the cannula and carefully move up and down and turn inside with slight pressure for at least 30 seconds. Each time before removing the brush, rinse the brush head under running water. Then brush the outside of the cannulas for a further 30 seconds. Avoid exerting excessive pressure and do not brush the cuff.
- Then soak the tracheal cannula and accessories in order to loosen any incrustation or clogging.

- To that end, pour a cleaning solution with the cannula cleaning powder according to the manufacturer's instructions in the cleaning tub with the sieve insert. The cleaning solution has to be enough to entirely cover the tracheal cannulas and accessories during the application time. Wear gloves while handling the cleaning solution in order to avoid skin contact with the cleaning solution.
- The cuff pilot must not be immersed in the cleaning solution!
- Place the tracheal cannula and accessories in the sieve insert of the cleaning tub and immerse in the cleaning solution without the cuff pilot, do not leave any air inside the cannulas. All parts must be completely immersed and soaked for at least 10 minutes (observe manufacturer instructions).
- Then carefully lift the sieve insert out of the cleaning solution and remove all parts.
- Clean the tracheal cannulas again with the cleaning brush in order to remove any remaining impurities and then rinse all parts repeatedly and thoroughly under clear running water in order to remove any residual cleaning solution.
- After cleaning, dry all parts with a lint-free cloth or leave to dry completely at room temperature.
- Dispose of the cleaning solution in accordance with manufacturer instructions.

7.4 Instructions for disinfection

- Wash hands.
- Inflate the cuff before disinfection.
- Pour a 2% disinfection solution with the cannula disinfectant according to the manufacturer's instructions in the cleaning tub with the sieve insert. The disinfection solution has to be enough to entirely cover the tracheal cannulas and accessories during the application time. Wear gloves while handling the disinfection solution in order to avoid skin contact with the disinfection solution. (Observe the manufacturer's instructions).
- The cuff pilot must not be immersed in the disinfection solution!
- Place the cleaned tracheal cannula and accessories in the sieve insert of the cleaning tub and immerse in the disinfection solution without the cuff pilot, do not leave any air inside the cannulas. The cannula must be completely immersed and soaked for at least 30 minutes (observe the manufacturer's instructions).
- Then carefully lift the sieve insert out of the disinfection solution and remove all parts.
- Rinse all parts repeatedly and thoroughly under clear, lukewarm running water in order to remove any residual disinfection solution.
- Then dry all parts with a lint-free cloth or leave to dry completely at room temperature.
- Dispose of the disinfection solution in accordance with the manufacturer's instructions.

Note

The suitability of the manual cleaning and disinfecting process described here has been confirmed by an independent, officially accredited testing laboratory using the SERVOX® cannula cleaning powder (concentration in accordance with manufacturer instructions, Dr. Schumacher GmbH) and the cannula disinfectant PRIMEDICLEAN (concentration in accordance with the manufacturer's instructions, Primed Halberstadt Medizintechnik GmbH).

8 Storage and transport

Store or transport clean and disinfected tracheal cannulas and accessories in a clean and dry environment, ideally in a lockable container free from dust and protected from light. Protect from direct sunlight and use, store or transport at temperatures between 5°C and 49°C.

For tracheal cannulas with cuff: Do not block the cuff during storage or transport.

9 Service life and disposal

In an intact sterile packaging and in compliance with the storage conditions tracheal cannulae and accessories can be stored for a maximum of 3 years. After opening the packaging, tracheal cannulae and accessories may be reused on one patient for a maximum of 29 days, when been cleaned and disinfected maximum 8 times by following the instructions of use. Damaged tracheal cannulae or accessories must not be used, but must be disposed of. Signs of damage include changes, discolouration or damage to the surfaces of the accessories, cannula or cuff, as well as dents or wrinkles in the cuff.

All tracheal cannulae and their accessories must be disposed of in accordance with national regulations for waste materials. If no special measures are necessary due to illness (e.g. MRSA), tracheal cannulae and your accessories can be disposed of with household waste.

EN

10 Technical data

All SERVVOX® Rota Tube variants have an angle of 100° and are made of latex and DEHP-PVC-free plastic and do not contain any metallic materials, there are no restrictions for use of magnetic resonance imaging (MRI).

The SERVVOX® Rota Tube cuff and SERVVOX® Rota Tube cuff voice variants are fitted with radiopaque strip on the supply tube to check the position.

10.1 Parameters of the SERVVOX® Rota Tube

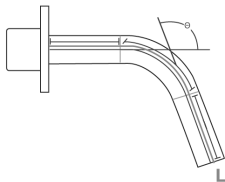
Figure 5: Diagram of the tracheal cannula

| Size | Outer diameter cannula* (in mm) | Outer diameter cannula OC* (in mm) | Length L*** (in mm) | Angle Θ (in °) | Cuff diameter **** (in mm) |
|------|---------------------------------|------------------------------------|---------------------|-----------------------|----------------------------|
| 6,5 | 9,0 | 6,5 | 60 | 100° | 22 |
| 7,0 | 9,6 | 7,0 | 65 | 100° | 23 |
| 7,5 | 10,3 | 7,5 | 70 | 100° | 24 |
| 8,0 | 11,0 | 8,0 | 76 | 100° | 26 |
| 8,5 | 11,6 | 8,5 | 82 | 100° | 28 |
| 9,0 | 12,3 | 9,0 | 89 | 100° | 30 |
| 10,0 | 13,3 | 10,0 | 97 | 100° | 33 |

* Outer diameter measured at the widest cross-section ** Inner diameter measured at the smallest inner lumen *** Length measured over the central arch **** Only for version with cuff

11 Order information, replacement parts and accessories

| | SERVOX® Rota Tube basic | SERVOX® Rota Tube voice | SERVOX® Rota Tube cuff | SERVOX® Rota Tube cuff voice |
|------|----------------------------|----------------------------|---------------------------|---------------------------------|
| Size | REF | REF | REF | REF |
| 6,5 | 25987 | 27452 | 25972 | 27445 |
| 7,0 | 25988 | 27453 | 25973 | 27446 |
| 7,5 | 25989 | 27454 | 25974 | 27447 |
| 8,0 | 25990 | 27455 | 25975 | 27448 |
| 8,5 | 25991 | 27456 | 25976 | 27449 |
| 9,0 | 25992 | 27457 | 25977 | 27450 |
| 10,0 | 25993 | 27458 | 25978 | 27451 |

Accessories

| Name | Order no. |
|----------------------------|-----------|
| SERVOX® Tracheo Tape M duo | 35207 |
| SERVOX® Remove Wedge RW15 | 29100 |

Accessories for use with the SERVOX® Rota Tube, such as cleaning and disinfection products, care sets, tracheal compresses and artificial noses are also available.

For additional information, please contact your field representative or Orbisana Healthcare GmbH.

12 Disclaimer

No repairs or changes may be made to the product by persons not authorised by Orbisana Healthcare GmbH. In the event of damage, the manufacturer does not assume liability in case of non-compliance with the instruction manual or the safety instructions or manipulation of the product.

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