SERVOX® Tracheo Tape

- DE** Einteiliges Kanülentrageband mit Haken (beige)
- EN** One-piece tube holder with hook (beige)
- FR** Bande porte-canule monobloc avec crochet (beige)
- IT** Nastro portacannula intero con gancio (Beige)
- ES** Cinta de fijación para cánulas, de una pieza, con gancho (beis)
- NL** Canuledraagband uit één stuk met haakjes (wit)
- EL** Μονοκύματη φέρουσα κορδέλα σωλήνα με γάντζο (μπεζ)
- BG** Поддържаща лента за канюли, от една част, с кукички (бежова)
- HR** Jednodijelna fiksacijska traka za kanile s kukicama (bež)
- PL** Opaska do kaniuli tracheostomijnych z haczykiem (kolor beżowy)
- PT** Fita de suporte de cânula com ganchos (bege)
- SL** Enodelni nosilni trak za kanilo s kavljii (bež barve)
- HU** Egyrészés kanültartó pánt kapocscsal (fehér)

REF XXXXX

LOT XXXXXX

PZN XXXXXXXX

JJ-MM

 (xx)XXXXXXXXXXXXXX

 (xx)XXXXXX

 (xx)XXXXXX

Orbisana Healthcare GmbH
 Biberweg 24-26 | 53842 Troisdorf | Germany
 Tel. +49 (0)2241 9322-0
 info@orbisana-hc.de | www.orbisana-hc.de

412580/Version 2022-11-28

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GB Intended use | indication | description

The SERVOX® Tracheo Tape is used in laryngectomised and tracheotomised patients at home and in a clinical setting together with tracheal cannulas or other aids with eyelets which keep the trachea open.

The SERVOX® Tracheo Tape secures the aid in the tracheostoma and prevents it from accidentally slipping out.

- Polyester tube holder
- Suitable for all age groups depending on the neck circumference
- single-use, do not reuse (risk of infection)
- The maximum duration of use is 48 hours, then dispose of in household waste
- Can be seamlessly adjusted to the circumference of the neck

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GB Application

- 1 Place the band around the back of the neck
- 2 Insert both hooks into the eyelets on the neck flange from below. The back of each hook is facing the throat.
- 3 Use the slider to adjust the length of the band.
- 4 Check fit. The band must be tight enough to ensure it does not accidentally come loose but not so tight that it constricts the throat!
- 5 To remove, unhook one side of the band from the neck flange.

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GB Please note

- Regularly check the fit and adjust where necessary.
- Remeasure the neck circumference regularly to ensure the correct size is always used.
- The gap between the padding and the neck flange should be as close as possible.
- Please inform us and, if necessary, the competent authority if life-threatening illness or injury occurs in connection with this product or if inpatient treatment results from its use.

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