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EN	Colour white	Material thermoplastic rubber, silicone	Length 54 cm for a legcircumference of 50 cm	Intended purpose For attaching urine leg bags to the upper or lower leg
----	------------------------	---	--	---

REF 25226

PZN 09095918

LOT XXXXXXXXX

YYYY-MM-DD

YYYY-MM-DD

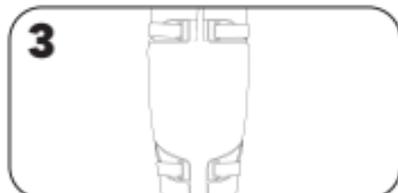
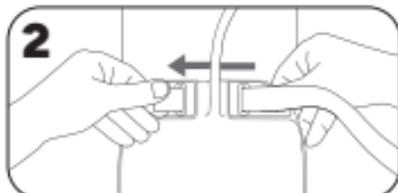
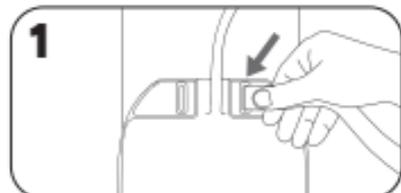
(xx)xxxxxxxxxxxxxxxx
(xx)xxxxxx
(xx)xxxxxx

Orbisana Healthcare GmbH
Biberweg 24-26
53842 Troisdorf | Germany
Tel. +49 (0)2241 9322-0
info@orbisana.de
www.orbisana.de

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Description:

With SERVOCURA® leg bag straps, urine leg bags with eyelets can be attached to the upper or lower leg. The straps are fixed with a Velcro fastener, the rubberized backside provides support on the skin.

EN

Warning: Do not tighten the straps too tight on the leg. Risk of thrombosis!

Instructions for use:

- 1:** Insert the strap with the rubberized backside to the leg from the front through the upper right pre-cut eyelet of the urine bag.
- 2:** Pull the strap end out through the upper left pre-cut eyelet, pass both strap ends around the leg and close with the Velcro fastener.
- 3:** Pull the second strap according to the first strap through the two lower pre-cut eyelets of the urine bag and close the strap.

Note: Close the Velcro fasteners for washing. Let dry completely before use. Dispose of the straps in domestic waste, if they are damaged or discolored, at the latest after 14 days.

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