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## EN INSTRUCTION FOR USE

### 1. Device description

The SERVOX® cannula cleaning brush is a cleaning brush designed specifically for cleaning tracheostomy tubes.

The wire used does not have any sharp edges and is made of stainless steel; the bristles are made of hygienic nylon.

The scope of supply includes four SERVOX® cannula cleaning brushes and one set of instructions for use.

### 2. Intended use

The SERVOX® cannula cleaning brushes are used specifically to clean tracheostomy tubes of various diameters.

### 3. Indications

Suitable for cleaning tracheostomy tubes with different diameters.

### 4. Contraindications

It is recommended to use alternative products to clean tracheostomy tubes with an inner diameter smaller than 5 mm.

### 5. Side effects

There are no known side effects if the indications, contraindications, warnings and instructions for use are considered and followed.

If you have any questions or in case some complications arise, please consult your physician.

### 6. Use

Bend the brush before use in accordance with the curve in the tube. Always remove the tracheostomy tube from the tracheostoma before cleaning. Always clean the tube from the tip (from the bottom end).

Slide the brush through the tube until the bristles are ticking out at the other end. Clean the bristles under running water before pulling them back through. Be careful during the cleaning procedure to avoid damaging the tube material.

After use, clean the brushes in accordance with the information provided in the section entitled "Cleaning, disinfection and care".

### 7. Instructions for use

- Inspect the tube before re-insertion to ensure that it is intact.
- The brush may be reprocessed a maximum of 21 times (e.g. 3 times a day for one week). After that, reprocessing is no longer recommended.
- The cleaning brush can be disposed of with household waste.
- The SERVOX® cannula cleaning brush is a single-patient-use product.

### Attention

- **Before using the brush for the first time, it must be cleaned and disinfected!**

### 8. Cleaning, disinfection and care

Required tools and accessories:

- Commercial dishwashing detergent (pH neutral, fragrance neutral).
- Suitable tray for cleaning and disinfection baths
- Clean, lint-free cloth
- Access to warm, clean running water
- Ethanol 70%

Cleaning instructions:

- Wash hands.
- Rinse the brush thoroughly under hand-hot running water. If necessary, rub off gross dirt from the brush and the ring handle at of the brush with a lint-free cloth under slight pressure.
- Prepare a cleaning solution of two drops of commercial dishwashing detergent with hot water (30 °C to 40 °C) in the tray.
- Immerse the brush in the cleaning solution with the brush head facing down and gently rotate it several times to ensure that no air remains in the spaces between the bristles, then immerse the brush completely and allow it to soak for at least 30 seconds.

- Then remove the brush.
- If contamination remains, repeat the cleaning procedure and then rinse the brush thoroughly several times under clear running water to remove any residue of the cleaning solution.
- Then let the brush dry completely at room temperature with the bristles facing up in a clean tray.
- Dispose the cleaning solution down the house drain.

Disinfection instructions:

- The brush must first be cleaned and then disinfected.
- Add a sufficient amount of ethanol 70% into the tray.
- Immerse the brush with the brush head down and gently rotate several times to ensure that no air remains in the spaces between the bristles, then immerse the brush completely: Soak for 5 minutes in ethanol 70%.
- Then remove the brush from the ethanol.
- Thoroughly rinse the brush on all sides under clear running water to safely remove any residues of the ethanol.
- Then let the brush to dry completely at room temperature with the bristles facing up and in a clean cup.
- Dispose of disinfectant solution down the house drain.

### 9. Warnings

- Always remove the tube from the tracheostoma before cleaning. Never use brushes on the tracheostoma.
- Clean and disinfect your brushes only with suitable and recommended cleaning agents or disinfectants. Do not use aggressive household agents, such as disinfectants that release chlorine or contain strong alkalis.
- Do not boil the SERVOX® tube cleaning brushes.
- We advise against the use of the product if it shows signs of damage such as sharp edges, tears or similar.
- The product must not be used if the packaging is damaged.

Failure to follow these instructions can have serious consequences for the health of the patient and may even result in life-threatening situations. If these instructions are ignored or if the product is manipulated, the manufacturer is unable to accept liability in the event of damage.

### 10. Safety

Each product is assigned a batch number, which is shown on the packaging, in order to ensure unambiguous traceability.

### 11. Storage and Transport:

Store or transport cleaned and disinfected brushes in a clean, dry environment, free of dust and protected from light.

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