SERVOX®

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Orbisana

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- EN SERVVOX® Tracheal Compress AL, with slit
- FR SERVVOX® Compresse trachéale AL, fendue
- IT SERVVOX® Compresa tracheale AL, fessura
- ES SERVVOX® Compresas traqueales AL, ranuradas
- BG SERVVOX® трахеални компреси AL, със срез
- HU SERVVOX® légcső-sebárna AL, bevágott
- EL επίθεμα τραχειοστομίας SERVVOX® AL, με σχισμή
- HR Trahealne komprese SERVVOX® AL, s prorezom
- NL SERVVOX® Tracheokompres AL met opening
- PT Compresas traqueais SERVVOX® AL ranhuradas
- PL Kompresy tracheostomijne SERVVOX® AL, z wycięciem
- SL Trahealne komprese z zarezo SERVVOX® AL

 **yyyy-mm**
PZN 08469512

 
 (xx)xxxxxxxxxxxxxx
 (xx)xxxxxxx
 (xx)xxxxxxx

Inhalt / Contents / Contenu
 Contenuto / Contenido
 съдържание / tartalmat
 περιεχόμενα / sadržaj / inhoud
 conteúdo / zawartość / vsebine

DE Nicht wiederverwenden – Infektionsgefahr
EN Do not reuse – risk of infection
FR Ne pas réutiliser – risque d'infection
IT Non riutilizzare – rischio di infezione
ES No reutilizar – riesgo de infección
BG Да не се използва повторно – опасност от инфекция
HU Ne használja újra – Fertőzésveszély
EL Μην επαναχρησιμοποιείτε – Υψηλότερα κίνδυνος μόλυνσης
HR Nemojte ponovno upotrebljavati – opasnost od infekcije
NL Niet hergebruiken – risico op infecties
PT Não reutilizar – Perigo de infeção
PL Nie używać ponownie – ryzyko infekcji
SL Izdelek ni primeren za ponovno uporabo – obstaja nevarnost okužbe

10 Stück
 pcs. | pcs. | pz. | броя
 darab | τεμάχιο | kom. | stuks
 unidades | szt. | kosov

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EN

Intended purpose SERVOX® Tracheal Compress AL, with slit, provide cushioning for the neck flange and absorb moisture that secretes from the tracheostoma to prevent skin irritation and infections.

Indication Tracheotomised or laryngectomised tracheal cannula users with normal secretions.

Description The aluminium coating helps prevent the compress from sticking to the skin. The slit allows the compress to be changed without removing the tracheal cannula.

Application Slide the tracheal compress with the aluminium coating to the tracheostoma through the slit over the cannula tube until the tube is fully in the punched hole of the compress and the neck flange is well cushioned. To remove, pull the compress off the cannula tube through the slit and dispose of it. – **Changing frequency:** with every cannula move frequently as needed, at least every 24 hours. – If the skin is irritated, use a compress with no aluminium coating and consult a doctor.

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