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EN

### 1. Intended use und description

The SERVOX® Neo tube cleaning brushes are used specifically to clean tracheostomy tubes with an inner diameter from 3-5 mm.

The wire of the brush is made of stainless steel without sharp edges, the bristles are made of nylon.  
**Scope of delivery:** 4 SERVOX® Neo cannula cleaning brushes, instructions for use.

#### Note

Only reuse the SERVOX® Neo cannula cleaning brush on a single patient, otherwise there is a risk of infection.

### 2. Indication

Patients using tracheal cannulae with an inner diameter of 3-5mm.

### 3. Contraindications

Patients using tracheal cannulae with an inner diameter larger than 5 mm or smaller than.

### 4. Sideeffects

No side effects known. If you have any questions or if complications arise, please consult your attending doctor.

### 5. Safety instructions

Not observing the safety instructions may result in serious health effects or even life-threatening injuries.

#### Warning

- Clean the tracheal cannula only outside the tracheostoma, never use the brush on the tracheostoma!
- Do not use any aggressive household agents that release chlorine or contain strong alkaline solutions to clean and disinfect the brush!
- Do not use the product if it is damaged, e.g. if it has sharp edges, tears, etc. or if the packaging is damaged!

#### Note

We ask you to inform us and, if applicable, the competent authority if you suffer a life-threatening illness or injury or permanent bodily harm in connection with the use of our product or if you have to be treated as an in-patient or prolong an in-patient treatment due to the use of our product.

## 6. Use

- Before first use and after each use, clean and disinfect according to the instructions for use.
- Pre-bend to fit the cannula bow before use.
- Clean the tracheal cannula under running water. To avoid damaging the cannula material, be careful. Always insert the brush from the lower end of the cannula through the cannula until the bristles are out at the other end, completely.
- Rinse the bristles under running water before withdrawing them.
- Always check the cannula for damage before reinserting it.

## 7. Cleaning and disinfection

The brush may be reprocessed a maximum of 7 times. Always wash hands before cleaning or disinfecting.

### Tools and accessories required:

- Commercial dishwashing detergent (pH neutral, fragrance neutral).
- Suitable container for cleaning and disinfection baths
- Clean, lint-free cloth
- Access to warm, clean running water
- Ethanol 70%

### Cleaning:

- Rinse the brush thoroughly under running, lukewarm water. If necessary, gently rub off gross dirt from the brush and ring handle with the cloth.
- Prepare a cleaning solution by mixing two drops of dishwashing detergent with warm water (30°C to 40 °C) in the container.
- Immerse the brush head in the cleaning solution and turn it carefully several times to remove remaining air between the bristles.
- Then immerse the brush completely and let it soak for at least 30 seconds.
- Repeat the process until no more contaminations are visible.
- Then rinse the brush thoroughly several times under clear running water to remove all residues of the cleaning solution.
- Allow brush to dry completely at room temperature with brush head upright in a clean cup.
- Dispose of cleaning solution down the house drain.

### Disinfection:

- Only disinfect cleaned brushes.
- Pour enough ethanol 70% into the container to cover the brush head completely.
- Immerse the brush head in the ethanol and gently rotate it several times to remove remaining air between the bristles.
- Then immerse the brush completely and let it soak for at least 5 minutes.
- Remove brush and rinse thoroughly under clear running water to rinse out all ethanol residues.
- Allow brush to dry completely at room temperature with brush head upright in a clean cup.
- Dispose of disinfectant solution down the house drain.

## 8. Storage, transport and disposal

Store or transport disinfected brushes dust-free, clean and dry at temperatures between 10°C and 25°C, protected from light. Dispose of with household waste.

## Order Information

Description	Unit	Order-no.
SERVOX® Neo tube cleaning brush	4	36554

## 9. Used Symbols

	CE-mark		Central pharmaceutical number
	medical device		Date of manufacture
	Single patient multiple use		Expiry date
	Consult instructions for use		Keep away from sunlight
	Do not use if is damaged		Temperature limit
	Manufacturer		Non-sterile
	Caution		Dual collection and recycling system
	Order number		
	Lot number		

EN

## 10. Disclaimer

In the event of non-compliance with the instructions for use or tampering with the product, the manufacturer accepts no liability in the event of damage

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**Servona GmbH**